# CAREO

MANUEL D'UTILISATION  
USER MANUAL

 **BMB  
MEDICAL**

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## FRANCAIS

---

### **BMB MEDICAL**

100 Avenue du Formans

CS 11001

01604 TRÉVOUX CEDEX

Tél : +33(0)4 74 08 71 71

Fax : +33(0)4 74 08 71 72

[www.bmb-medical.com](http://www.bmb-medical.com)

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Fax : +33(0)4 74 08 71 72

[www.bmb-medical.com](http://www.bmb-medical.com)

The purpose of this manual is to inform you how to use, maintain and ensure the safety of your new medical device. We greatly advise that you read this document carefully and that you follow the recommendations for an optimum use in safe conditions.

**BMB MEDICAL** offers a wide variety of options for the stretcher-trolley referred to in this User Manual. As a result, some of the functionalities described herein may not be applicable to your stretcher-trolley.

Some of the presented illustrations are for indication purposes and are not an exact representation of the device that you have in your possession.

The information contained in this document can be modified without warning and BMB MEDICAL shall not be held responsible in any way. Any representation or reproduction, whether total or partial, carried out without BMB MEDICAL's authorisation is illicit. This illicit representation or reproduction, by any process, would be considered as counterfeiting. The illustrations included in this document are not contractual.

## CONTENTS

ENGLISH .....	22
1 INTRODUCTION SPECIFICATIONS.....	25
1.1 PRODUCT PRESENTATION .....	25
1.1.1 CAREO STRETCHER TROLLEY .....	26
1.2 OBJECTIVE OF THIS USER MANUAL.....	27
1.3 USERS .....	27
1.4 STORAGE.....	27
1.5 SYMBOLS AND STANDARDS USED .....	28
2 INSTRUCTION FOR USE.....	29
2.1 INSTALLATION PROCEDURE .....	29
2.2 HEIGHT ADJUSTMENT AND LOWERING .....	29
2.2.1 RAISING THE STRETCHER .....	29
2.2.2 LOWERING THE STRETCHER.....	30
2.3 BACKREST .....	30
2.4 SIDERAILS.....	30
2.5 BRAKING AND GUIDING PEDAL .....	31
2.6 BACKARD / FORWARD TILTING SYSTEM.....	32
2.7 OPTIONS .....	33
2.7.1 RETRACTABLE PUSHBARS .....	33
2.7.2 REMOVABLE IV POLE .....	33
2.7.3 VERTICAL OXYGEN BOTTLE SUPPORT SYSTEM.....	34
2.7.4 CHART HOLDER FOR PAPERWORK UNDER THE HEADREST .....	34
2.7.5 BAG HOLDER HOOKS .....	34
3 CLEANING AND DISINFECTION .....	35
3.1 INTRODUCTION .....	35
3.2 FREQUENCY .....	35
3.3 CLEANING AND DISINFECTION ADVICE.....	35
3.3.1 CLEANING OPERATION .....	35
3.3.2 RECOMMENDED EQUIPMENT AND PRODUCTS.....	36
4 PRECAUTIONS.....	37
4.1 DEVICE-RELATED PRECAUTIONS .....	37
4.1.1 WARNINGS .....	37
4.1.2 CONTRAINDICATIONS .....	38
4.1.3 PRECAUTION ASSOCIATED WITH EQUIPMENT CARE AND MAINTENANCE .....	38

4.2	TRANSPORT ET STOCKAGE.....	39
4.2.1	TRANSPORT .....	39
4.2.2	STORAGE .....	39
4.3	WARRANTY .....	39
4.4	AFTERSALES DEPARTMENT .....	40
4.5	ENVIRONMENT .....	40
4.5.1	WORN EQUIPMENT .....	40

# 1 INTRODUCTION SPECIFICATIONS

---

## 1.1 PRODUCT PRESENTATION

This patient support device consists of a patient surface fitted onto a chassis with wheels. The device is equipped with side bars and, optionally, allows the temporary fitting of IV poles.

The device allows an operator to convey the patient within a healthcare establishment. The 5th wheel option allows you to switch from straight in-line guidance during transport to the ability to pivot the product around a corner. The stretcher is braked by a mechanical system actuated by pressing a pedal.

The device is equipped with a hydraulic cylinder system to ensure the movements:

- of the backrest,
- the 2 columns allowing the height adjustment of the product.

The products are designed to ensure that there are no unacceptable risks (essential performance) associated with patient support or caused by accidental movements.

### 1.1.1 CAREO STRETCHER TROLLEY

<b>TECHNICAL CHARACTERISTICS</b>	
<b>REFERENCE</b>	CAREO
<b>DESIGNATION</b>	HEIGHT-ADJUSTABLE HYDRAULIC STRETCHER-TROLLEY
<b>SAFE WORKING LOAD</b>	200 kg (maximum acceptable load including the patient's total weight, mattress and fittings)
<b>WEIGHT</b>	80 kg
<b>OVERALL DIMENSIONS</b>	68 x 205 cm
<b>TRENDELENBURG / REVERSE TRENDELENBURG</b>	+/- 18 °
<b>BACKREST</b>	0 to 90 °
<b>EQUIPEMENTS</b>	<p>2-sectional mattress base: 60 x 190 cm.            Thermoformed ABS base.            Height adjustment range: from 65 to 100 cm (including 8cm thick mattress).            Trendelenburg / Anti-Trendelenburg +/-18°, hands free control.            Removable mattress made of high quality and resilience foam (35kg/m<sup>3</sup>), height : 8 cm.            Removable cover. PVC coated Jersey. Colour : grey.            2 holders for IV pole head-end.            Bumpers on all 4 corners.            Backrest adjustable by 2 gas springs (0 - 90°).            Castors Ø 200 mm with central steering and braking system foot-end.            2 directional wheels head-end.            Drop-down side rails, 2 horizontal bars. Anodized aluminium.            Fixed push bar foot-end.            Frame cover including O<sub>2</sub> tank holder.            Maximum patient weight: 135 kg.</p>
<b>AVAILABLE OPTIONS</b>	<p>5th directional wheel.            Removable IV pole, 2-stage, 2 hooks.            Removable IV pole, 4-stage, 2 hooks.            Foldable push bars foot-end.            Foldable push bars head-end.            Vertical oxygen bottle holder.            Paper roll dispenser at the head-end.            Chart holder (for paperwork) below the headrest.            Attachment hooks for bags on both sides below the patient surface.            Comfort mattress.            Side rail padded cushions (pair).            Universal Meal tray / Extensible meal tray for side rails: from 65 to 88 cm / Fall protection side.</p>

\* BMB MEDICAL can in no way be held responsible for any potential problems caused if the mattress being used does not correspond to the mattress reference recommended by BMB MEDICAL.

## APPLICATION FIELD

This stretcher trolley is intended according to its specifications, optional features and accessories for use in stretcher services, day surgery and hospital emergency units.

It is designed for transporting, treating and transferring patients and for their rest and recovery.

### 1.2 OBJECTIVE OF THIS USER MANUAL

This operating manual is intended for health professionals in order to furnish you with all the necessary information and all of precautions to be followed to ensure effective and safe use of this appliance.

The instructions relating to the maintenance and inspection of the product should be followed to ensure satisfactory equipment performance.

It is therefore essential that all staff have access to these instructions and are aware of the handling procedures and risks associated with the use of the stretcher trolley. The device must be handled by a qualified person who has been made aware of the risks associated with using the device incorrectly.

### 1.3 USERS

Healthcare professionals: physicians, caregivers and agents approved by Technical Services who are accustomed to working with similar equipment can operate all functions.

### 1.4 STORAGE

This user manual should be kept near the product, in a place provided for this purpose. It must be shielded from all elements and substance that could compromise its perfect legibility.

## 1.5 SYMBOLS AND STANDARDS USED

This stretcher is a Class I non-invasive and active medical device.

	See the manual / instruction brochure
	Indicates the maximum patient weight and safe working load
	Conforms to the essential requirements of directive 93/42/EEC on medical devices.
	Serial number
	Manufacture date
	Manufacturer
	Maximum patient weight
	Protéger de l'humidité / keep dry
	Haut, tenir debout / stand up
	Fragile / fragile
	Recyclable / recycling

General safety sign:

- Information highlighted by this symbol must be read and scrupulously observed.
- Indicates that failure to observe the guideline could endanger the patient or user, or damage the equipment.

Sitting on the backrest is prohibited

Sitting on the legrest is prohibited

The chair must be set to its low comfort (or lying) position before being moved.

**“WARNING: This appliance must not be altered without the manufacturer’s written consent.”**

**“Caution: only purpose-built accessories provided for this MD by BMB MEDICAL may be used.”**

## 2 INSTRUCTION FOR USE

---

### 2.1 INSTALLATION PROCEDURE

Before commissioning the chair and depending on the model, check that the following are operational:

- [Height adjustment and lowering](#)
- [Backrest](#)
- [Siderails](#)
- [Braking and guiding pedal](#)
- [Backward/ forward tilting system](#)

### 2.2 HEIGHT ADJUSTMENT AND LOWERING

Before using these functions:

**Make sure all persons and objects are clear of the area under the stretcher top,**

**Make sure that nobody's feet are placed under any of the pedals (height adjustment, trendelenburg/reverse trendelenburg).**

#### 2.2.1 RAISING THE STRETCHER

The control system (grey pedal) is located at the patient's feet.


The bed is raised by pumping the pedal :

## 2.2.2 LOWERING THE STRETCHER

The bed is lowered by pressing simultaneously on the two backward (yellow pedal) and forward (blue pedal) tilting pedals.

Make sure that your foot is placed so as to apply a homogeneous pressure on the 2 pedals.

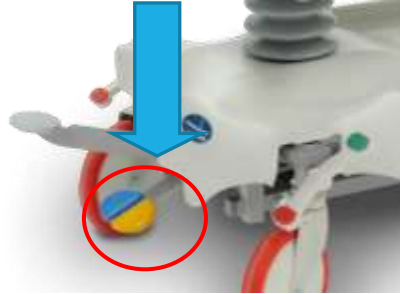
## 2.3 BACKREST

**Before operating the backrest lifting system, check that nothing is blocking it (fittings, cables, etc.).**

The backrest lifting system, assisted by 2 gas springs, is controlled by operating the bar located under the lying surface: apply a pressure (with 2 hands to make it easier) to unblock the cylinders and adjust the backrest lifting system at the chosen height.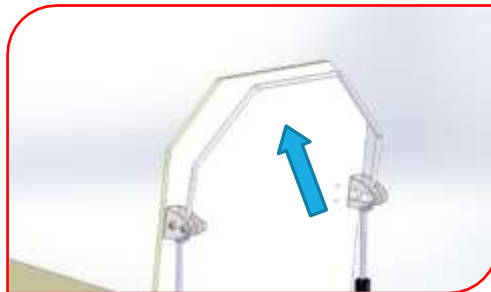

## 2.4 SIDERAILS


**The bars are provided to reduce the risk of the patient accidentally falling from the stretcher. They are in no way designed to immobilise or maintain the patient with strap-type immobilisation devices.**

**Before handling the bars, make sure that nothing is blocking their movement.**


### **Lowering the bars :**

Le To unlock, maintain the button pressed down ① and push the bar ②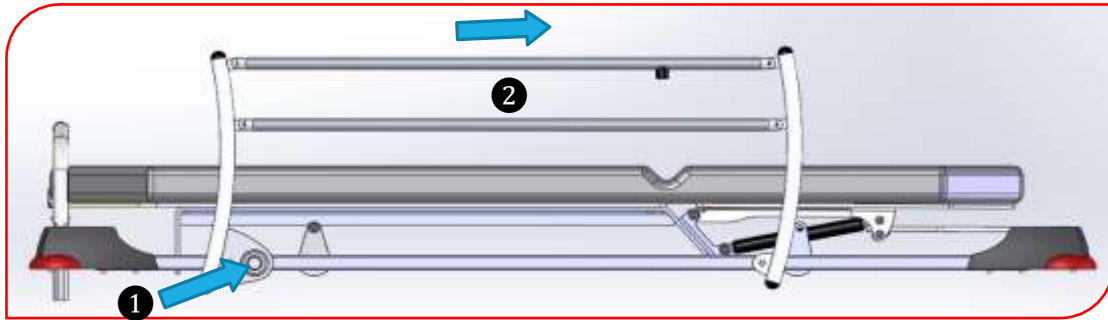

### **Lifting the bars :**

Lift the top bar until the button is locked.

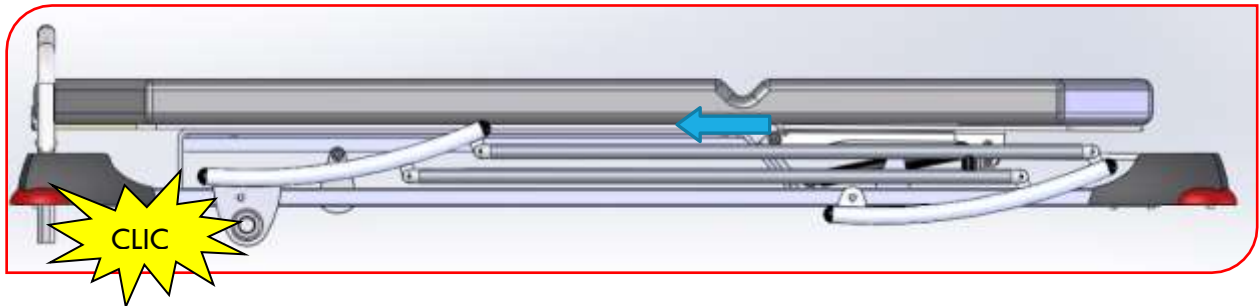## **2.5 BRAKING AND GUIDING PEDAL**

**Except during transport, make sure that you always apply the stretcher-trolley's brake. Check that the brake is active by trying to move the stretcher-trolley.**

**Do not park the stretcher-trolley on a slope.**

The braking and guiding pedal is used for 3 functions:

- **Locked stretcher-trolley function** - red side pedal tilted downwards ①.
- **Unlocked stretcher-trolley function**, which is used to move it in several directions on 4 wheels (5th wheel is deactivated) - pedal in horizontal position ②.
- **Stretcher-trolley in directional guiding** mode which makes it easier to move the trolley in a straight line - side green pedal tilted downwards ③.

## 2.6 BACKARD / FORWARD TILTING SYSTEM

### Trendelenburg Position:

In order to place the stretcher-trolley in the backward tilt position (stretcher head in low position), press on the yellow pedal.

### Reverse Trendelenburg Position:

In order to place the stretcher-trolley in the forward tilt position (stretcher foot in low position), press on the blue pedal.


## 2.7 OPTIONS

### 2.7.1 RETRACTABLE PUSHBARS

In order to retract the handles, lift the locking ring up to the stop, the handles are then free to rotate downwards, away from the lying surface.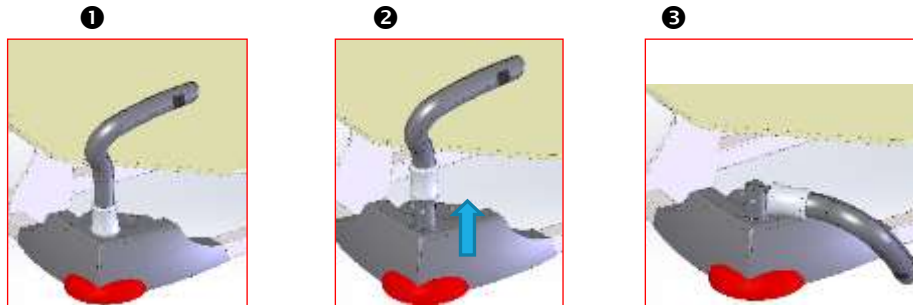

In order to reposition the handles, lift them and replace the ring in its low position. Check that they are positioned correctly.

### 2.7.2 REMOVABLE IV POLE

The removable drip stand is placed in one of the two fitting supports provided for this purpose.

To deploy the IV pole:

- 1- Turn the thumb wheel ❶ counter-clockwise and extend the telescopic part of the IV pole ❷ to the desired height.
- 2- To lock the telescopic part, turn the thumb wheel ❶ clockwise. Ensure that the IV pole is securely locked.

**Safe working load: 6 kg.**


**This indicative value corresponds to safe product use, but is not representative of normal use for which the load is significantly lower.**

**When moving the stretcher fitted with the IV pole, ensure that the latter does not strike any obstacles (e.g. door frame, lighting, etc.) or individuals.**


**Do not move the stretcher by the IV pole.**

### 2.7.3 VERTICAL OXYGEN BOTTLE SUPPORT SYSTEM

Rotate the oxygen bottle support system away from the lying surface.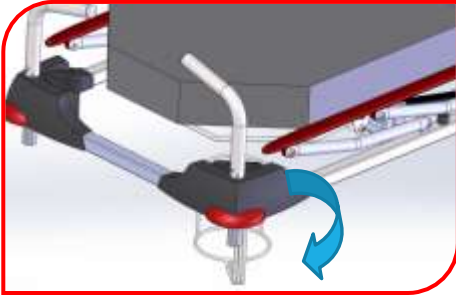


Retracted bottle support system

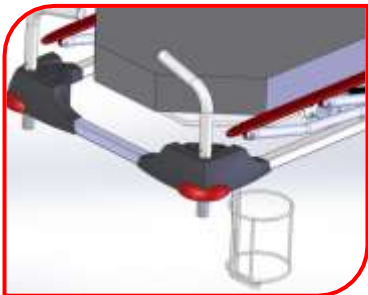
Extended bottle support system

### 2.7.4 CHART HOLDER FOR PAPERWORK UNDER THE HEADREST
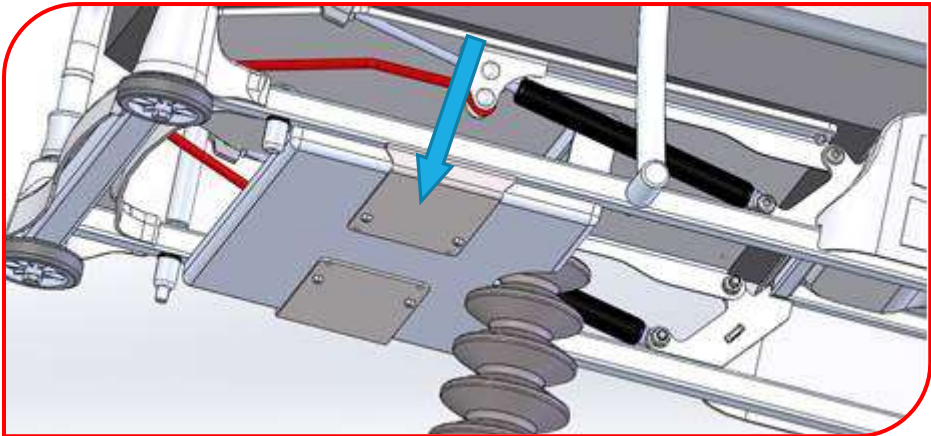
### 2.7.5 BAG HOLDER HOOKS
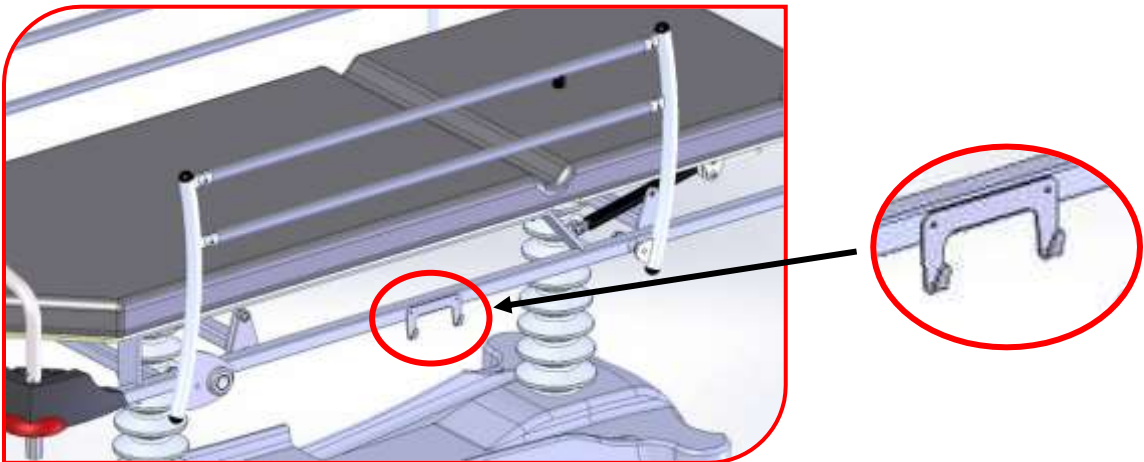
## 3 CLEANING AND DISINFECTION

---

### 3.1 INTRODUCTION

Regular and correct cleaning ensure the product's life service (see cleaning program prepared upon commissioning of the device by appointing the reference personnel).

**If the cleaning operations are not carried out, this may cause a risk of infection due to the presence of secretions and/or residues on the stretcher-trolley.**

The following cleaning recommendations do not replace your establishment's internal protocol.

### 3.2 FREQUENCY

The cleaning shall be performed:

- After each patient i.e. between two patients.
- After transporting an infected patient.
- Entirely on a weekly basis.

### 3.3 CLEANING AND DISINFECTION ADVICE

#### 3.3.1 CLEANING OPERATION

For routine cleaning of the upholstery, painted stainless steel or aluminum surfaces, plastic parts and control elements, a gentle detergent such as soapy water may be used, followed by rinsing with clear water. After cleaning, dry the product thoroughly.

All of the product's surfaces must be cleaned.

During cleaning or disinfection, it is recommended to:

- Spray or apply the cleaning product to the MD with a clean disposable cloth.
- Start with the areas closest to the patient, focusing on parts in contact with users.
- Proceed from top to bottom.
- Allow the equipment to dry.

Products that cause stains (e.g.: "blue product" or eosin) must be cleaned as rapidly as possible to prevent them from soaking in.

Aerial disinfection of surfaces is allowed for this product.

### 3.3.2 RECOMMENDED EQUIPMENT AND PRODUCTS

Use only authorized and standardized cleaning products or disinfectants: quaternary cleaning products, phenolic compounds and bleach solutions, etc.

- Observe the prescribed dosages and dilutions.
- Observe the application conditions (contact time, quantity applied, rinsing, etc.)
- Do not mix the products.
- Ensure that all disinfectant detergents used are compatible with:
  - o PVC, ABS, Polyamide, Polyurethane and Polypropylene
  - o Epoxy paint-coated metal surfaces
  - o Stainless steel and aluminum surfaces
- Familiarize yourself with the products strictly prohibited by the manufacturer (see list below).

#### List of strictly prohibited products:

Solvents:	Gasoline / aliphatic and aromatic higher alcohols / ketones-esters / trichloroethylene-perchloroethylene
Bases:	10% Sodium hydroxide / 10% Ammonia
Acids:	10% Nitric / 30% Hydrochloric / 10% Citric
Miscellaneous:	Pure or weakly diluted bleach / Abrasive powders / All abrasive products

NB:

- **Quaternary cleaning products** consist of ingredients containing “-yl” suffix elements and ammonium chloride
- **Quat./isopropyl cleaning products** consist of a quaternary ingredient, as described above, along with isopropyl alcohol
- **Phenolic cleaning products** consist of ingredients containing the “-pheno” suffix
- **Bleach** is known under the generic name “sodium hypochlorite”

Never use abrasive utensils.

Use clean disposable cloths.

Wear protective clothing and gloves during cleaning and disinfection.

**High-pressure and steam cleaning are prohibited.**

**Any damage resulting from failure to observe this instruction for use of a detergent-disinfectant will not be covered by BMB MEDICAL under the terms of the warranty.**

## 4 PRECAUTIONS

---

### 4.1 DEVICE-RELATED PRECAUTIONS

#### 4.1.1 WARNINGS

**This MD has been designed for and adapted to medical practices and procedures, and hence a specific use. Abnormal use of the latter can lead to user or patient accidents, or even to irreversible damage.**

The following list of abnormal chair uses is not exhaustive and BMB MEDICAL counts on the common sense of the chair's users.

- Do not use the product without first having familiarized yourself with this user manual.
- Do not alter the stretcher. Any changes made to the system could lead to unpredictable operation that could in turn jeopardize the safety of the user or patient. This would also void the chair's warranty.
- Used to accommodate a person of small stature (less than 1.30 m tall), or under the age of 12 years without adapted equipment (pediatric kit)
- To ensure correct use of the product, only proceed if the weight is correctly distributed over the device.
- The chassis cover load capacity is of 35 kg. Do not sit or stand on the cover.
- Only purpose-built accessories provided for this MD by BMB MEDICAL may be used
- Only activate the various positions after having ensured that no-one is touching any mechanical parts and that there are no objects under the seat or patient surface, or between any of the moving structures.
- The side rails are intended to minimize the risk of the patient accidentally falling. They are not, in any way, designed to immobilize or maintain the patient.
- Before manipulating the side rails, ensure that nothing could hinder their movement.
- When lowering the side rail, take care not to trap the hands or feet of the patient or staff
- If the side rail locking operation is silent (no "click"), please revise the side rails.
- Except during transport, always ensure that the chair's brake is engaged. Check whether the brake is engaged by attempting to move the chair.
- Patient sitting and standing operations should be performed, with the stretcher in the low position, from the side, after lowering the side rails.
- The "variable height" function should be used when handling the patient in order to work at an ergonomic height.
- Do not park the chair on a sloping floor.
- Use outside of the establishment, or on unprepared ground.
- Do not use the MD on a floor with a slope of more than 10%.
- Use of the chair with a load that exceeds the safe working load (see technical characteristics).

- When moving the stretcher fitted with the IV pole, ensure that the latter does not strike any obstacles (e.g. door frame, lighting, etc.) or individuals.
- Do not move the stretcher via the IV pole
- In the case of an unsupervised patient, the product should be left in its lowest position to reduce the risk of falling.
- Tell the patient not to sit on the ends (foot or head) of the chair/stretcher. An excess weight applied to one of the ends could tip the patient surface and injure the patient.
- Failure to perform these cleaning operations can lead to a risk of infection caused by the presence of secretions and/or residues on the stretcher.
- High-pressure and steam cleaning are prohibited.
- Any use that does not fall within the operating context of a hospital stretcher.
- Use by any non-caregiver who has not received the authorization and explanations from a qualified individual.

**All manipulations relating to the aforementioned precautions for use are described in [paragraph II - Instructions for use](#).**

#### **4.1.2 CONTRAINDICATIONS**

There are no noteworthy contraindications for this device.

#### **4.1.3 PRECAUTION ASSOCIATED WITH EQUIPMENT CARE AND MAINTENANCE**

The device must not be altered in any way without the manufacturer's prior consent: the alteration could lead to unpredictable operation, potentially injuring the patient and caregivers.

Torn upholstery no longer presents an effective antibacterial barrier and must hence be replaced as soon as possible.

The device must be regularly inspected, at least once per year. See the maintenance check-list. To maintain the device in good condition and to guarantee its safe and durable operation, the prescribed maintenance must be performed

In the event of observed anomalies or damage that could jeopardize device operation and safety, the equipment must be immediately withdrawn.

The device should only be used according to the instructions in this user manual.

The device must not be exposed to, or even less be in contact with, sources of heat, combustion and flammable agents.

\* BMB MEDICAL will accept no liability in the event of incidents or accidents resulting from improper use or alteration of the product by the buyer or user.

## 4.2 TRANSPORT ET STOCKAGE

### 4.2.1 TRANSPORT

Before transporting the stretcher with a view to storing to ensure that there are no risks of impact or accidental falls during transport, we recommend that you:

- Fasten stretcher to the frame
- Place the cover over it
- Lock the brakes
- Place it in the lowest position

Damage to the appliance caused during transport is not covered by the guarantee. Repairs or replacement of damaged parts are the responsibility of the client.

### 4.2.2 STORAGE

When the chair is in storage, ensure that:

- Its brakes are engaged
- It is covered

The product should be stored in a clean, ventilated, temperate and dry place, protected from the elements and from direct sunlight.

During storage, no heavy weights should be placed on the chair.

The chair must not be considered as a support surface for any other type of equipment.

The device must not be exposed to, or even less be in contact with, sources of heat, combustion and flammable agents.

## 4.3 WARRANTY

Warranty: two years parts and labor according to the following terms:

*BMB MEDICAL grants its customers a commercial warranty of two (2) years from the date of delivery, within the limits defined below.*

*This commercial warranty is not intended to replace the legal guarantee of conformity mentioned in articles L.211-4 to L.211-13 of the French Consumer code, or that pertaining defects of the purchased object, covered by articles 1641 to 1648 and 2232 of the French Code of civil law.*

*Beyond the legal guarantee, BMB MEDICAL guarantees the exchange and replacement of parts accepted to be out of order, due to machining fault, assembly fault or material failure. The scope of the warranty does not include parts made unusable either by normal wear, or by improper use or non-compliant repair.*

Where applicable, at the request of BMB MEDICAL, any product or part that is the subject of a warranty claim must be returned postage paid to the BMB MEDICAL plant. Any improper use, alteration or repair performed with parts not supplied or certified by BMB MEDICAL will void this warranty.

BMB MEDICAL stretchers are designed for a projected service life of 10 years under normal conditions of use, with appropriate periodic maintenance as described in the maintenance check-list of each stretcher. BMB MEDICAL provides the original buyer with the guarantee that the welds on the stretchers are free from structural defects for the projected stretcher service life of 10 years, as long as the original buyer is in possession of the product.

No guarantees are provided concerning appropriateness for a specific use.

This warranty excludes any disposable parts, IV poles, mattresses, batteries, along with damage incurred by abuse.

## **4.4 AFTERSALES DEPARTMENT**

For all information concerning the installation, use and maintenance of the product and any other questions, please contact the after-sales department at the following address: sav@bmb-medical.com or phone +33 (0)4 74 08 71 71 or contact your local distributor if you are located abroad.

Thanks to our "Client Space", you can access your equipment online, see the maintenance operations to be performed and request our services directly via this secured interface.

The operator shall carefully read the information contained in this User Manual, especially that concerning the precautions of use for safety purposes and the installation and operating methods.

If you have any doubt regarding the correct interpretation of the instructions, do not hesitate to contact the after-sales department, which will always be available to provide advice.

In order to facilitate the ordering of spare parts, you can ask the after-sales department to send you an exploded view of your products. You will therefore be able to obtain the references of the parts you need.

## **4.5 ENVIRONMENT**

### **4.5.1 WORN EQUIPMENT**

Components reaching the end of their service life shall enter the appropriate recovery circuit (drop-centre) to be recycled.

The stretcher-trolley is easy to disassemble and can therefore be evacuated and recycled according to applicable regulations.

Tél : +33 (0)4 74 08 71 71  
Fax : +33 (0)4 74 08 71 72  
[info@bmb-medical.com](mailto:info@bmb-medical.com)

100 Av. du Formans – CS 11001  
01604 TREVOUX Cedex  
FRANCE  
[www.bmb-medical.com](http://www.bmb-medical.com)