


# CLAUIA STRETCHER BED

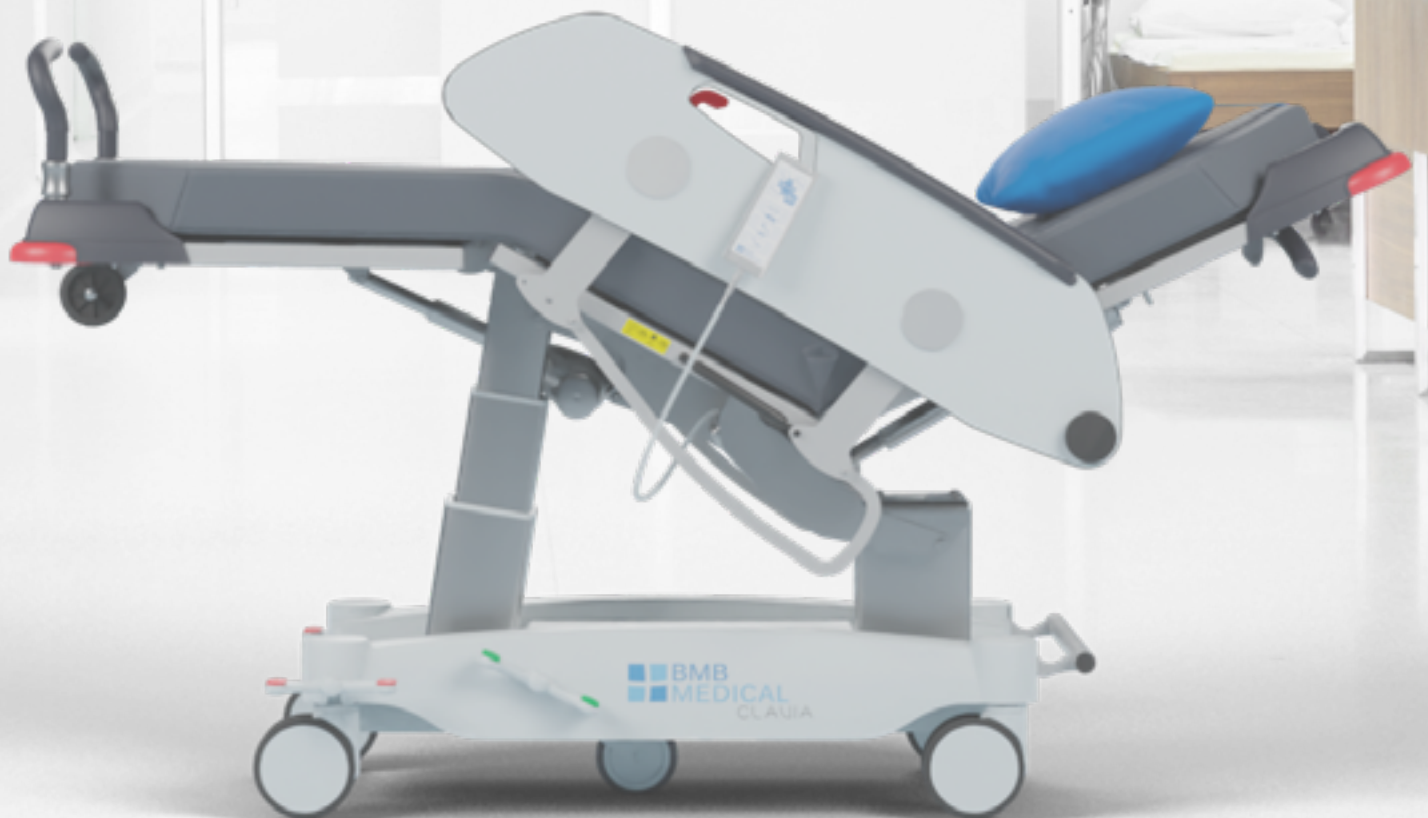

Our primary objective is your **satisfaction**. Our **requirements** are as exacting as we would expect as a customer.

By supporting you from defining the product and throughout its life, we guarantee **high-performance** products, and a **responsive** and **high-quality service**.

We try, with humility, to **understand** your **use**, your **constraints** and to redefine the circuit of your patients to **improve** your **working conditions** and the **efficiency** of your **organization**.

We base our relationships on **shared trust**, making it our priority to be by your side for the **long term**.

# UNDERSTAND YOUR USE, YOUR CONSTRAINTS

## An essential component of day care & day surgery

From the surgery version to the chemotherapy version, CLAVIA replaces the bed in outpatient wards and day hospitals. Opt for the CLAVIA LSA SURGERY interchangeable surgical headrests, gutter armrests and the comfortable CLAVIA FCD trim, or the compactness and flexibility of the CLAVIA LSA.

## Mobility

This very compact stretcher bed is fitted with a 5th directional wheel and diameter 150 mm double running wheels. Intuitive and easy steering requiring considerably reduced pushing efforts. No matter the weight, the chair is light and easy to handle.

## Reliability

Our products, submitted to the LCIE Bureau Veritas laboratory, have been CE certified. They conform to ISO 14971, EN 60601-1, EN 60601-1-2 and EN 60601-1-6 standards. They are tested and secure. [Find out more](#)

## Retractable handles

Their ergonomic shape make pushing easy and comfortable. Made from ultra-strong plastic, they don't rust. Under the backrest, the patient is stretched in the Comfort Position.



## Ultra comfort

One button on the remote control to reach zero gravity. Immediate feeling of well-being, the Ultra Comfort position balances the body's pressure on the bed.

## 3-position barriers

Ensuring patient safety when in the high position, they offer two low positions: the first is dedicated to transfers, the barriers completely disappear under the lying surface. The second is devoted to surgical procedures, the patient's chest is free and the surgeon is free to move around.

## Stability



With its two columns, the reliability of the CLAVIA is such that it can be used in the operating theater. Tested according to standard 60601-1, its high stability is certified.

## All electric

From variable height to pre-programmed positions, the chair is electrically guided by its user. Offering comfort, autonomy and safety to the patient, in addition to being easy to use for the staff, it is quickly adopted by the day care department.

# M O B I L I T Y

Our chairs have been designed to make your daily life easier

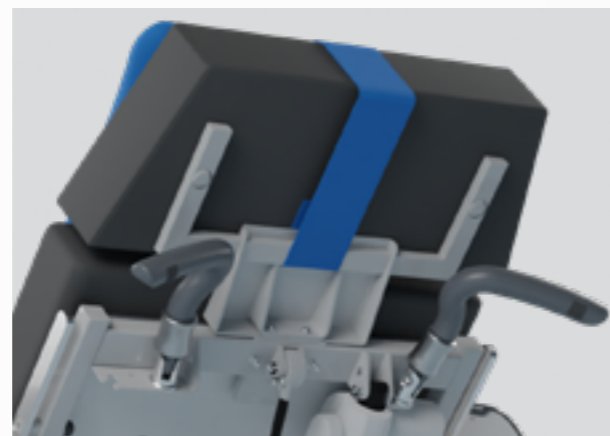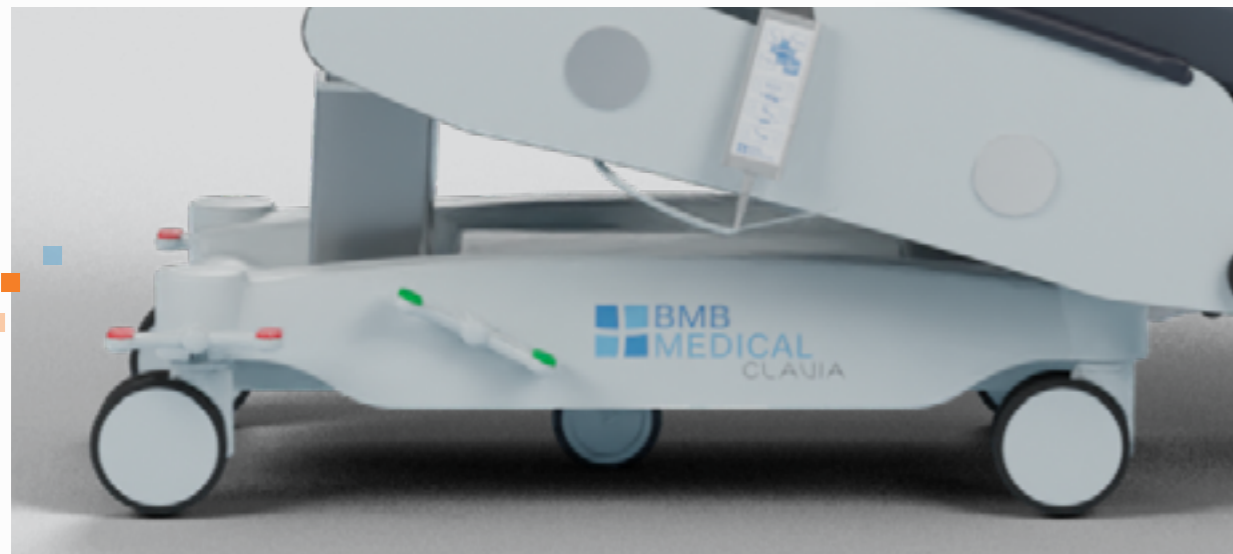

We understand that your time is precious. Each second counts. That is why our chair is so easy to use. Its fifth directional wheel makes it light and intuitive. Its compactness simplifies your movements. Its double runner wheels adapt to any floor or obstacle. Its push handles, placed under the backrest for more comfort, perfectly fit the shape of your hands. Improving your working conditions is one of our priorities.

Guaranteeing the **safety** of your patients



Patient safety is not optional. Our products have been tested to meet the EN 60601-1 standard: jamming, stability, maneuverability, braking efficiency, etc. The patient can safely be left alone. In an emergency, the Trendelenburg or Reverse Trendelenburg or CPR (cardiopulmonary resuscitation) positions can be reached within seconds.



# RELIABILITY

High-quality components,  
Long-term use,  
A reduced environmental footprint



Our environmental awareness has led us towards an eco-friendly design. BMB MEDICAL is concerned about the ecological impact of its products at each stage of their life cycle. This commitment is reflected in actions on packaging, the "4 R" policy (Reduce, Replace, Recycle, Rethink) and in our efforts on materials. Our products are designed and approved in accordance with standard IEC60601-1-9 that defines the requirements for eco-friendly design.



Our products are tested with constraints that are significantly higher than those it will be subjected to in daily use: 533 kg on the backrest, 924 kg on the lying surface, etc. Our columns are maintenance-free. Our sleeping surfaces are made from an extremely resistant and rot-proof material. Our corner protections are resistant to shocks and aging. Choosing BMB MEDICAL is a guarantee of quality all while offering significant savings on your maintenance budget.


# FOCUS ON DAY SURGERY



The CLAVIA LSA SURGERY has been designed for day surgery taking into account the surgeon's comfort. In the surgery position, the guards completely free the backrest, enabling the surgeon to operate as close as possible to the patient. They are placed at the correct height to avoid interference during endoscopy. The push bars retract to leave the field free for the surgeon. The foot pedal is used to adjust the height of the chair, even when in sterile clothing. The magnetic cushions and headrests are interchangeable depending on the type of procedure.

# DAY HOSPITAL

Choose relaxation, choose CLAVIA

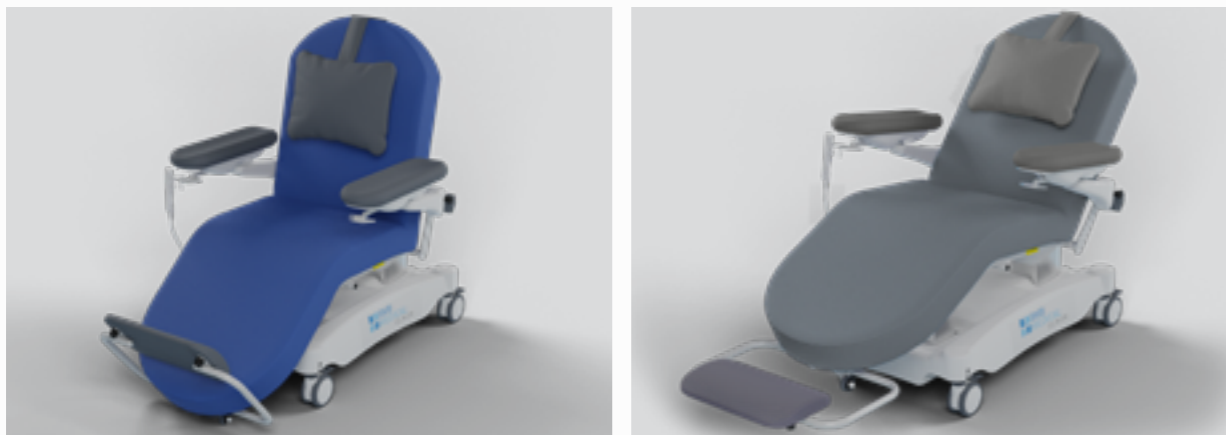

More compact, but as comfortable as a bed, the fully electric CLAVIA FCD chair offers rest and relaxation to the patient. Independent thanks to its remote control, the patient can use pre-programmed positions to settle down before and during their treatment. By distributing body pressure evenly over the mattress, the Ultra Comfort position provides immediate well-being. The footrest can be adjusted to limit cramps. The 360° multi-directional armrest conforms to the shape of the arm, follows the inclination of the backrest and is height-adjustable. It is replaced by guards for the FSP version.



## CONTENTS


▶ CLAVIA LSA CHIR

▶ CLAVIA LSA  
LSA KIDS

▶ CLAVIA FCD

▶ CLAVIA FSP

**Compact**, it replaces a bed by optimizing space  
**Ultra Comfort Position**, it reduces patient stress  
**Mobile**, it is equipped with a 5th directional wheel and push bars under the backrest  
**Stable**, it ensures patient safety

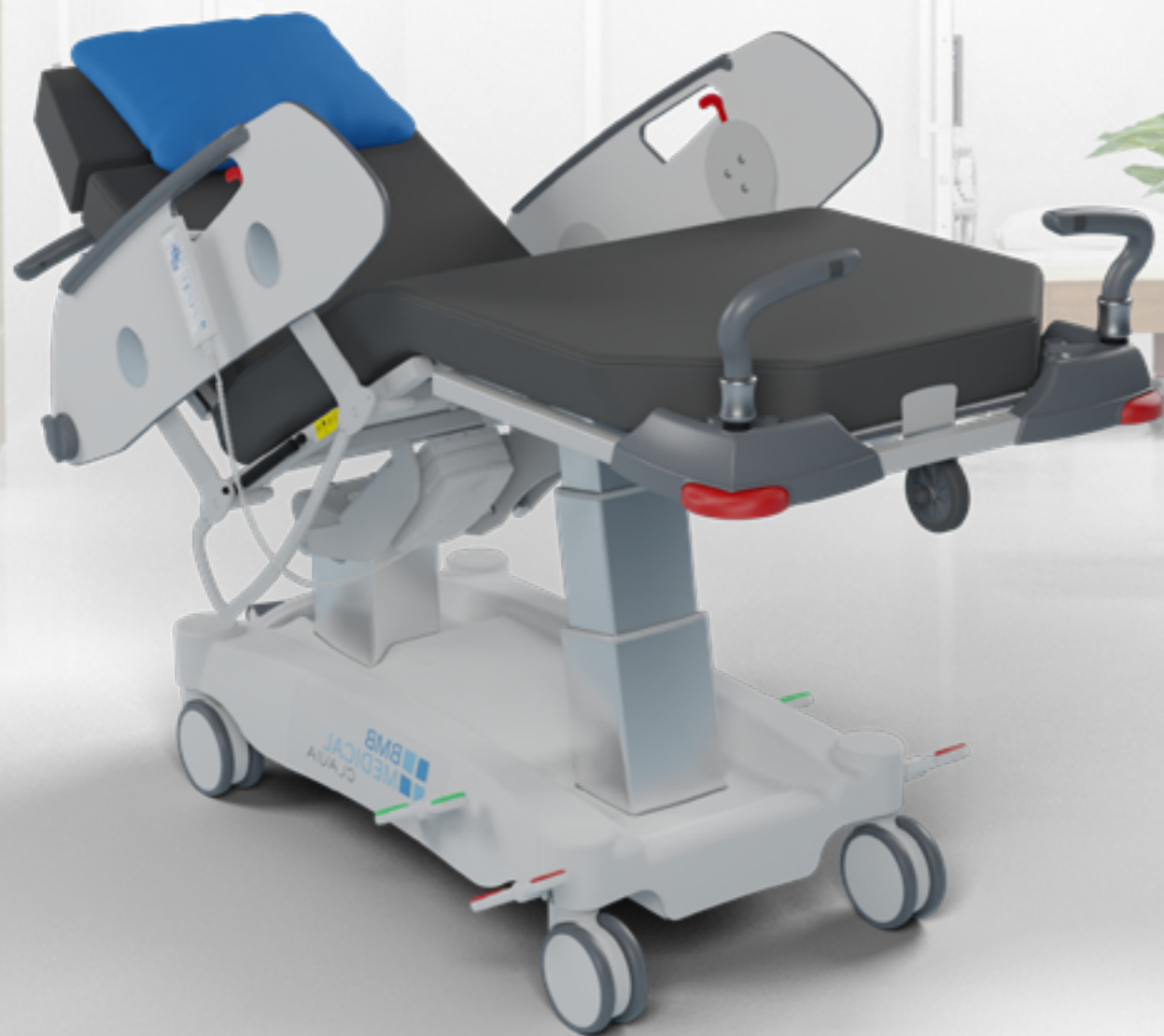

## STRETCHER BED **CLAVIA LSA SURGERY**

CLAVIA LSA SURGERY fits perfectly into the fast-track patient circuit, or directly into the day surgery unit.

From installation to the moment they leave the room, the patient no longer needs to be transferred. The procedure is performed directly on the chair.

Its remarkable mobility ensures good conditions of use for nursing staff.


CLAVIA LSA SURGERY is intended for ophthalmology, stomatology, ENT, hand surgery, colonoscopy, gastroscopy, etc.

# CLAVIA LSA SURGERY

## CLAVIA LSA SURGERY DIMENSIONS

	LENGTH	WIDTH	HEIGHT
OVERALL	152 cm chair position 213 cm bed position 215 cm bed position + headrest ophthalmology	80 or 90 cm (depending on model)	59 to 99 cm (with mattress)
MATTRESS BASE	65 x 200 cm, 75 x 200 cm		

PRODUCT WEIGHT : 113 - 116 Kg (depending on model)  
SAFE WORKING LOAD : 200 Kg

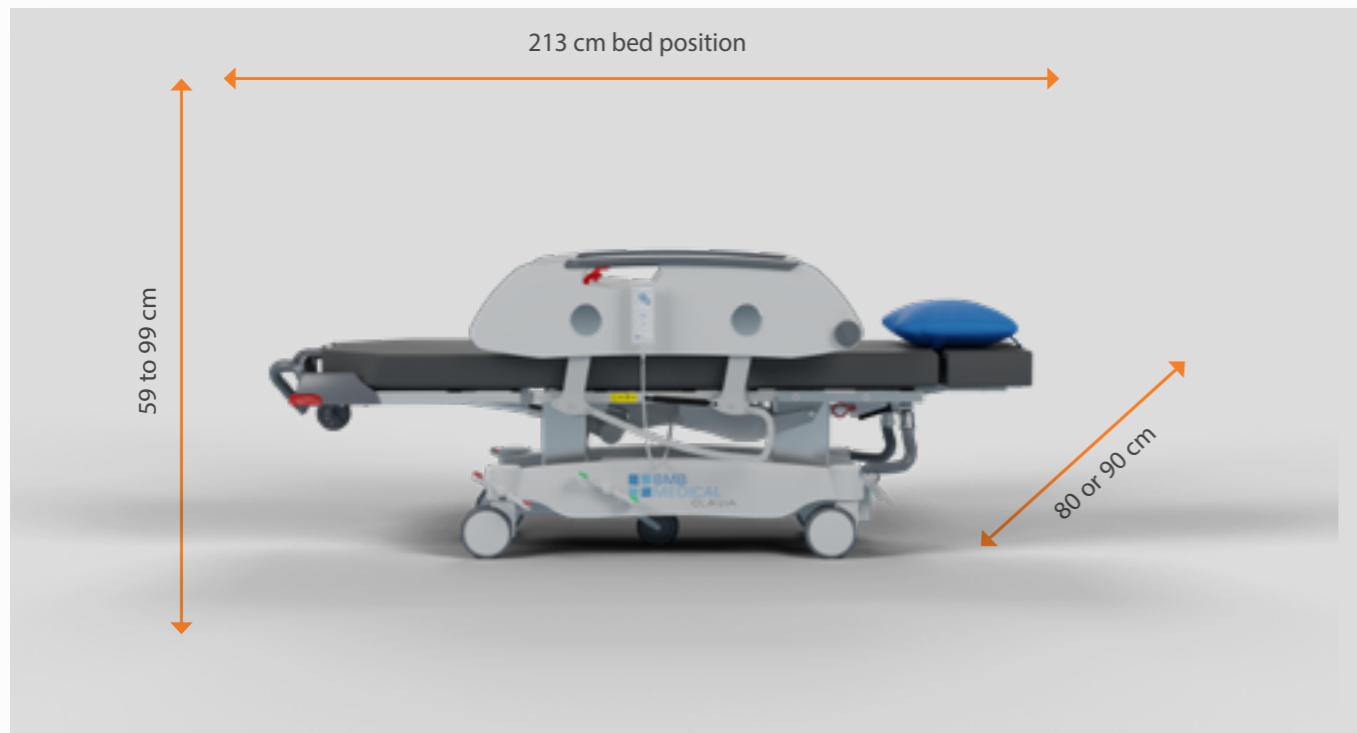

NON-CONTRACTUAL PHOTOS - PRODUCTS FITTED WITH ACCESSORIES

CLAVIA LSA SURGERY fits perfectly into the patient circuit in fast-track or directly in the surgery unit.

Combining comfort, ergonomics and aesthetics, our products are designed to accommodate and meet the needs of patients and caregivers.

### Stretcher bed for day surgery

- Electrically assisted height adjustment on 2 LINAK columns : 49 - 89 cm (without mattress 10 cm thick)
- Electric Trendelenburg (15°). Reverse Trendelenburg (6°)
- Electrically (LINAK) assisted leg rest 0 to 60°
- Electrically (LINAK) assisted backrest, manual release 0 to 90°
- Removable mattress made of high resilience foam (35 kg/m<sup>3</sup>) - Thickness 10 cm
- 2 section backrest with a removable head section to fasten a surgery head rest for ophthalmology, stomatology and ENT
- Digital actuators allowing 6 programmed positions: Welcome, Relax, Reverse Trend, Secured Trendelenburg position, CPR
- Heavy duty design. Steel frame with powder coating
- ABS bonnet
- Double running wheels with central braking system Ø 150 mm including 5th directional wheel
- Drop-down laminated side guards. Length 110 x Height 27cm
- 3 positions side rails - High security position - Lox position for easy access - Surgery position (ENT, Stomatology, Hand surgery, colonoscopy)
- Bumpers on 2 corners at foot end fitted with holder for accessories
- 2 holders on siderails for IV pole
- Technical rails on backrest at both sides
- Foldable push bars below backrest
- Battery Li-on 2.25A (50 cycles)
- Removable cover of PVC coated Jersey - Colour: slate grey



2 section backrest with a removable head section to fasten a surgery head rest for ophthalmology, stomatology and ENT.



Electrically Trendelenburg (15°)  
Reverse Trendelenburg (6°)



Double running wheels with central braking system Ø 150 mm including 5th directional wheel



3 positions side rails




6 programmed positions :  
Welcome, Relax, Reverse Trend, Secured Trendelenburg position, CPR






Anti-static system  
CLORA

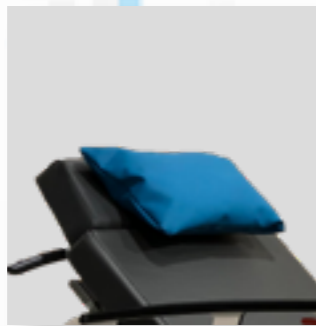

Comfort pillow (blue)  
ACC-CTB




Without the standard  
headrest (depreciation)  
CLOST

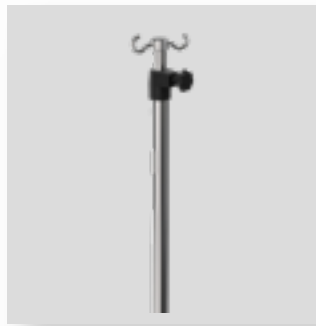

Removable IV pole, 2-stage, 2  
hooks  
TSR2




Holder for oxygen cylinder  
(incompatible with the option  
CLOEC)  
CLOSO2

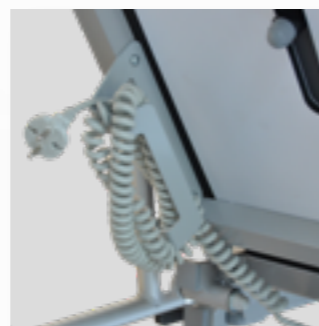

Cable reel (incompatible with the  
option CLOSO2)  
CLOEC



Articulated removable  
surgery head rest for oph-  
thalmology  
+ hollow cushion  
LCA-OPH  
Additional hollow cushion  
LCA-CC

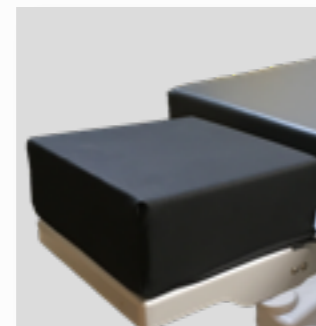

Articulated removable  
surgery head rest for sto-  
matology +  
flat cushion  
LCA-CP  
Additional flat cushion  
LCA-CP



Removable surgery head rest for  
stomatology and ENT  
LCA-STO



Attachment hooks for bags on  
both sides below the patient  
surface  
PSU



CLAVIA side rails cushion/ar-  
mrest, removable and right/left  
reversible (single)  
CLOCB



Support for meal tray  
CLOSPR-G  
CLOSPR-D

## ACCESSORIES

REFERENCE	DESCRIPTION
CLORA	Anti-static system
ACC-CTB	comfort pillow (blue)
CLOST	Without the standard headrest (depreciation)
TSR2	Removable 2-stage IV pole, 2 hooks
TSR4	Removable IV pole, 2-stage, 4 hooks
CLOSO2	Holder for oxygen cylinder (incompatible with the option CLOEC)
CLOEC	Cable reel (incompatible with the option CLOSO2)
CROBPP	Foldable push bars foot-end
LCA-OPH	Articulated removable surgery head rest for ophthalmology + hollow cushion
LCA-PLA	Articulated removable surgery head rest for stomatology + flat cushion
LCA-STO	Removable surgery head rest for stomatology and ENT
LCA-CC	Additional hollow cushion for articulated removable surgery headrest
LCA-CP	Additional flat cushion for articulated removable surgery headrest
LCA-SUP	Head rest support. Fixation on technical rails.
CLORD	Width of the backrest 60 cm for CLAVIA LSA CHIR 75 only.
LSA-PROTECT	Mattress protection at foot end
PSU	Attachment hooks for bags on both sides below the patient surface
EQUI	Equipotential connection
CLOHVP	Foot pedal control for variable height (in addition of the wired remote control)
CLOCB	CLAVIA side rails cushion/armrest, removable and right/left reversible (single)
CLOSPR-G	Support for meal tray on the left side of the patient (compatible with CLOSPR-D)
CLOSPR-D	Support for meal tray on the right side of the patient (compatible with the CLOSPR-G)
CLOPR-P	Small Shelf : 42 x 32 cm (CLOSPR-G or CLOSPR-D required)
CLOPR-G	Large Shelf : 81 x 41 cm (CLOSPR-G or CLOSPR-D required)
FSC	Holder for operating field, with connection for oxygen

**Ultra Comfort Position,** reduces patient stress

**Mobile,** it is equipped with a 5th directional wheel and push bars under the backrest

**Compact,** it replaces a bed by optimizing space

**Intuitive** with 6 pre-programmed positions

**Stable** it ensures patient safety



## STRETCHER BED **CLAVIA LSA**

The CLAVIA LSA is suitable for patient management in day surgery departments.

All its movements are electric.

Comfortable and easy to handle, it can replace a bed in an outpatient department by optimizing space.


Equipped with 6 pre-programmed positions, it is quick and intuitive to use.

A wide range of accessories are available to adapt it to any type of use.

## CLAVIA LSA DIMENSIONS

	LENGTH	WIDTH	HEIGHT
OVERALL	160 cm chair position 218 cm bed position	80 or 90 cm (depending on model)	59 to 99 cm (with mattress)
MATTRESS BASE	65 x 200 cm or 75 x 200 cm		

PRODUCT WEIGHT : 84 - 86 Kg (depending on model)  
SAFE WORKING LOAD: 250 kg

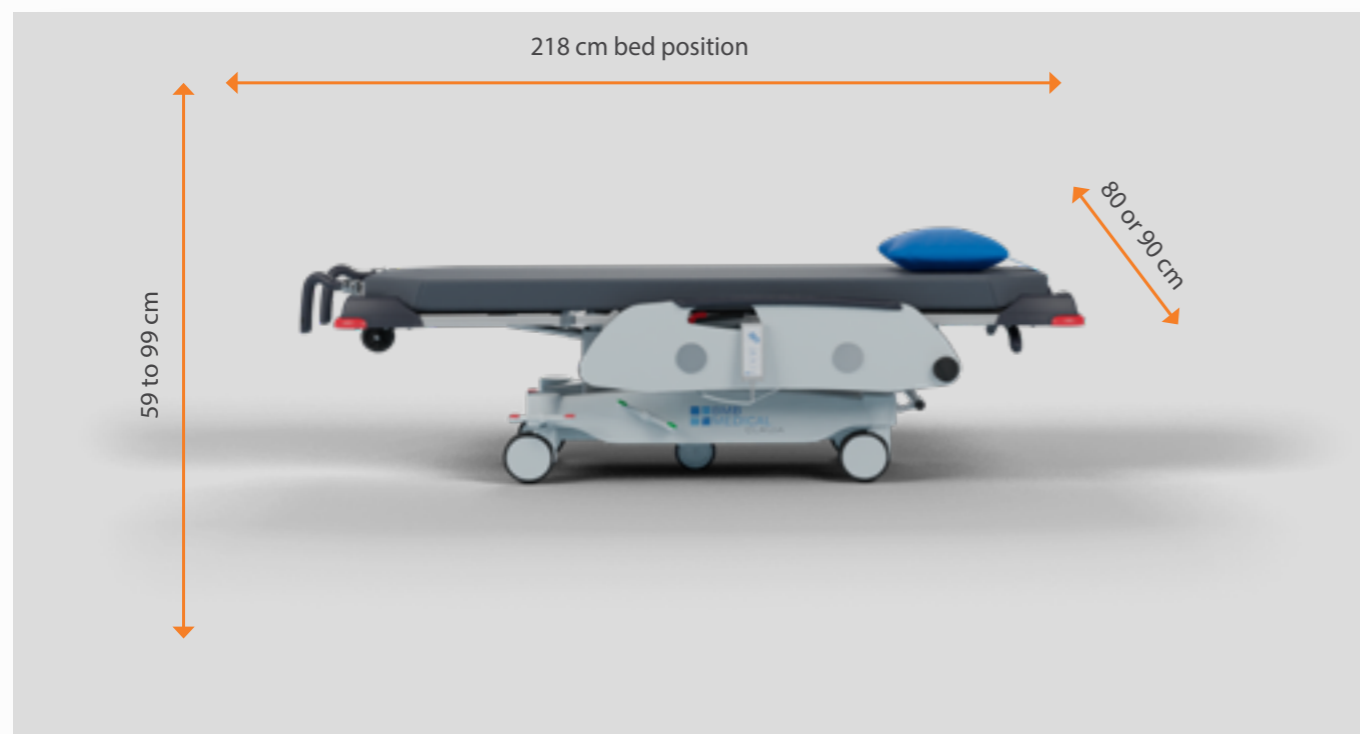

NON-CONTRACTUAL PHOTOS - PRODUCTS FITTED WITH ACCESSORIES

The CLAVIA LSA is suitable for patient management in outpatient departments.

Combining comfort, ergonomics and aesthetics, our products are designed to meet the needs of patients and caregivers.

### Stretcher bed for day care

- Electrically assisted height adjustment on 2 LINAK columns : 49 - 89 cm (without mattress 10 cm thick)
- ElectricTrendelenburg (15°) - Reverse Trendelenburg (6°)
- Electrically (LINAK) assisted leg rest 0 - 60°
- Electrically (LINAK) assisted backrest, manual release
- Removable mattress high resilience foam (35 kg/m3) - Thickness 10 cm
- Digital actuators allowing 6 programmed positions : Welcome, Relax, Reverse Trend, Secured Trendelenburg position, CPR
- Machine-welded steel frame, epoxy coated
- Frame protection cover
- Double running wheels with central braking system Ø 150 mm including 5th directional wheel
- Drop-down laminated side guards - Length 110 x Height 27cm above mattress
- 3 positions side rails - High security position - Lox position for easy access - Surgery position (ENT, Stomatology, Hand surgery, colonoscopy)
- Bumpers on all 4 corners fitted with holder for accessories
- 2 holders on siderails for IV pole
- Foldable push bars below backrest
- Battery
- Removable cover of PVC coated Jersey - Colour : slate grey



6 programmed positions :  
Welcome, Relax, Reverse Trend, Secured Trendelenburg position, CPR



Electrically Trendelenburg (15°)  
Reverse Trendelenburg (6°)



Double running wheels with central braking system  
Ø 150 mm including 5th directional wheel



Foldable push bars below backrest



3 positions side rails


# ACCESSORIES



Anti-static system  
CLORA




Comfort pillow (blue)  
ACC-CTB

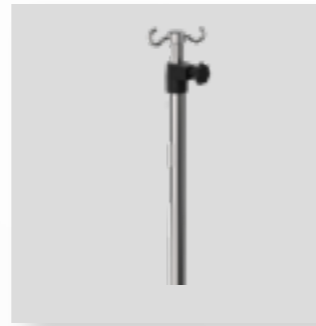

Removable IV pole, 2-stage, 2  
hooks  
TSR2



Holder for oxygen cylinder  
(incompatible with the option  
CLOEC)  
CLOSO2

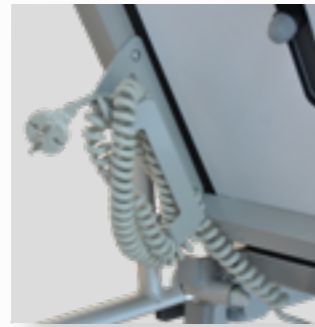

Cable reel (incompatible  
with the option CLOSO2)  
CLOEC



Foldable push bars head-end and  
foot-end CROBPT CROBPP



Mattress protection at foot end  
LSA PROTEC



High Comfort mattress, high quality  
foam (35kg/m3), 3 cm thick viscoelastic  
layer (total height : 13cm)  
MMM



Paper roll dispenser at the head-  
end (incompatible with BPSRB)  
CROSP



Attachment hooks for bags on  
both sides below the patient sur-  
face  
PSU



CLAVIA side rails cushion/armrest,  
removable and right/left rever-  
sible (single)  
CLOCB



Support for meal tray  
CLOSPR-G  
CLOSPR-D

REFERENCE	DESCRIPTION
CLORA	Anti-static system
ACC-CTB	comfort pillow (blue)
TSR2	Removable 2-stage IV pole, 2 hooks
TSR4	Removable IV pole, 2-stage, 4 hooks
CLOSO2	Holder for oxygen cylinder
CLOEC	Cable reel (incompatible with the option CLOSO2)
CROBPT	Foldable push bars head-end
CROBPP	Foldable push bars foot-end
LSA-PROTECT	Mattress protection at foot end
MMM	High Comfort mattress, high quality foam (35 kg/m3) + 3 cm thick viscoelastic layer (total height : 13 cm)
CROSP	Paper roll dispenser at the head-end
PSU	Attachment hooks for bags on both sides below the patient surface
EQUI	Equipotential connection
CLOSPR-G	Support for meal tray on the left side of the patient (compatible with CLOSPR-D)
CLOSPR-D	Support for meal tray on the right side of the patient (compatible with the CLOSPR-G)
CLOPR-P	Small Shelf : 42 x 32 cm (CLOSPR-G or CLOSPR-D required)
CLOPR-G	Large Shelf : 81 x 41 cm (CLOSPR-G or CLOSPR-D required)
CLOCB	CLAVIA side rails cushion/armrest, removable and right/left reversible (single)



# CLAVIA LSA KIDS PEDIATRIC STRETCHER BED



CLAVIA KIDS is a specific pediatric stretcher bed for day care.

CLAVIA KIDS complies, with its specific side rails and pediatric kit at head-end, with the requirements from April, 26th, 2010 of the French Medical Safety Agency ANSM with regard to children under 146 cm.

Fully electric, CLAVIA KIDS enables everyday medical staff using. It permits space optimization by replacing medical bed in day care units.

Very comfortable and with a playful design, CLAVIA KIDS will bring psychological comfort to children.

REFERENCE	DESCRIPTION
CLOBP	Polycarbonate pediatric side rails (instead of the standard side rails)
CLOKT	Pediatric kit head-end (only available for width 65 cm) - CLOKP included
CLOKP	Pre-equipment for pediatric kit (only available in width 65 cm)
CLOKCL	Lateral protection cushion for pediatric side rails
CLOKCT	Head protection cushion (CLOKCL obligatory)

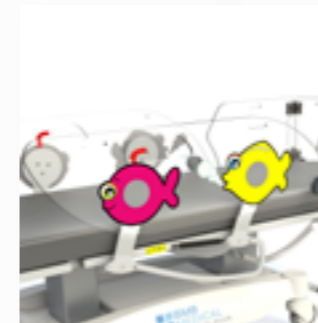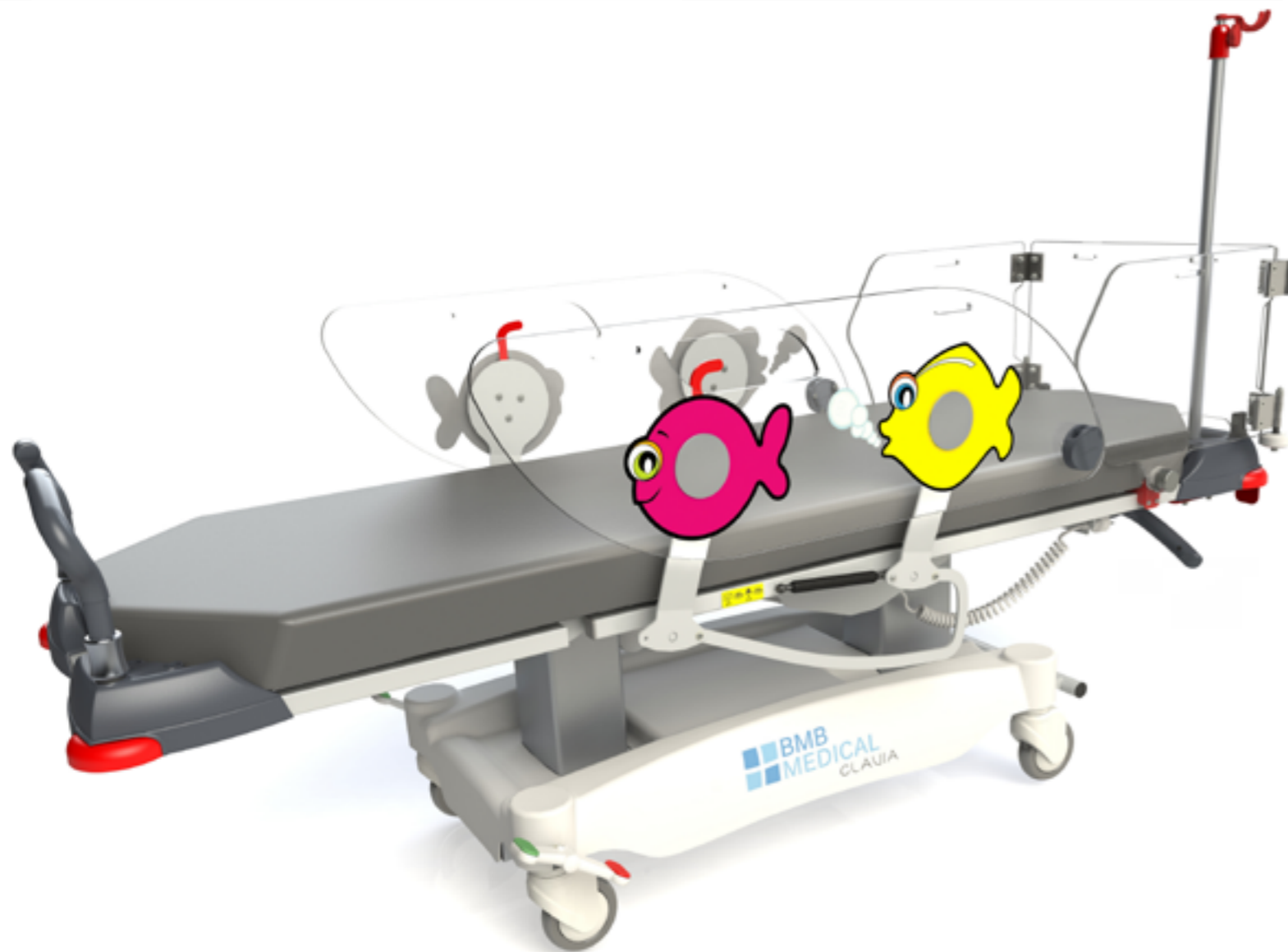

Polycarbonate pediatric side rails (instead of the standard side rails)




Pediatric kit head-end CLOKT



Lateral protection cushion for pediatric side rails and Head protection cushion CLOKCL & CLOKCT


Pediatric kit head-end CLOKT compatible with standard barriers

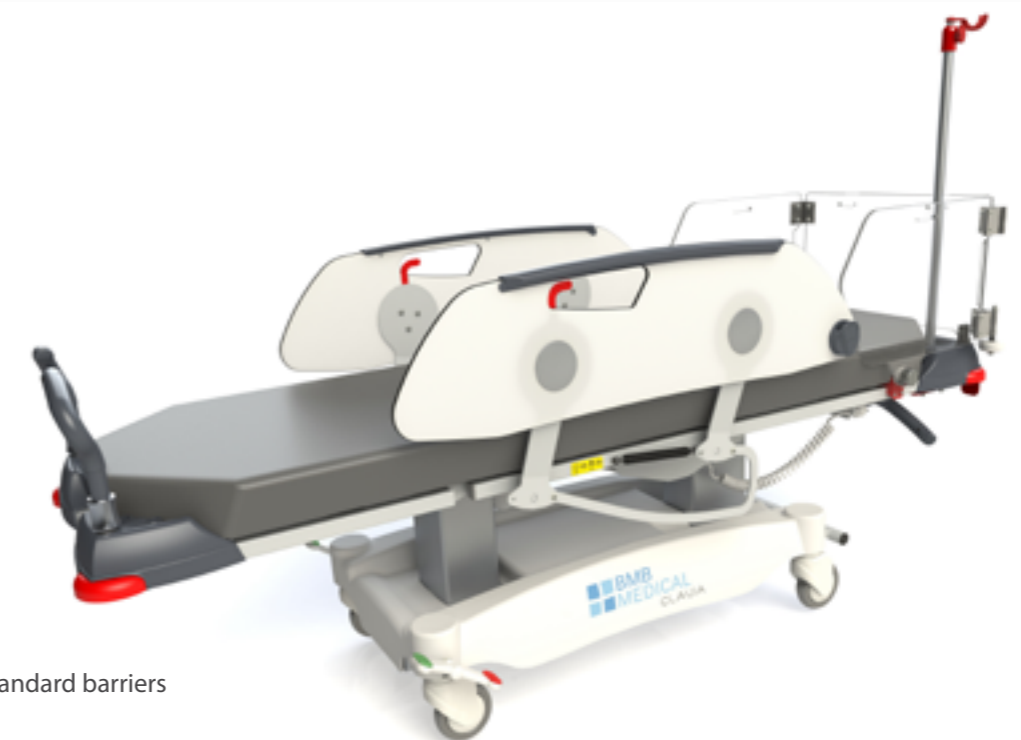

**Comfortable,** it has an ultra comfort position and a bed position

**Compact,** it replaces a bed by optimizing space

**Stable** it ensures patient safety

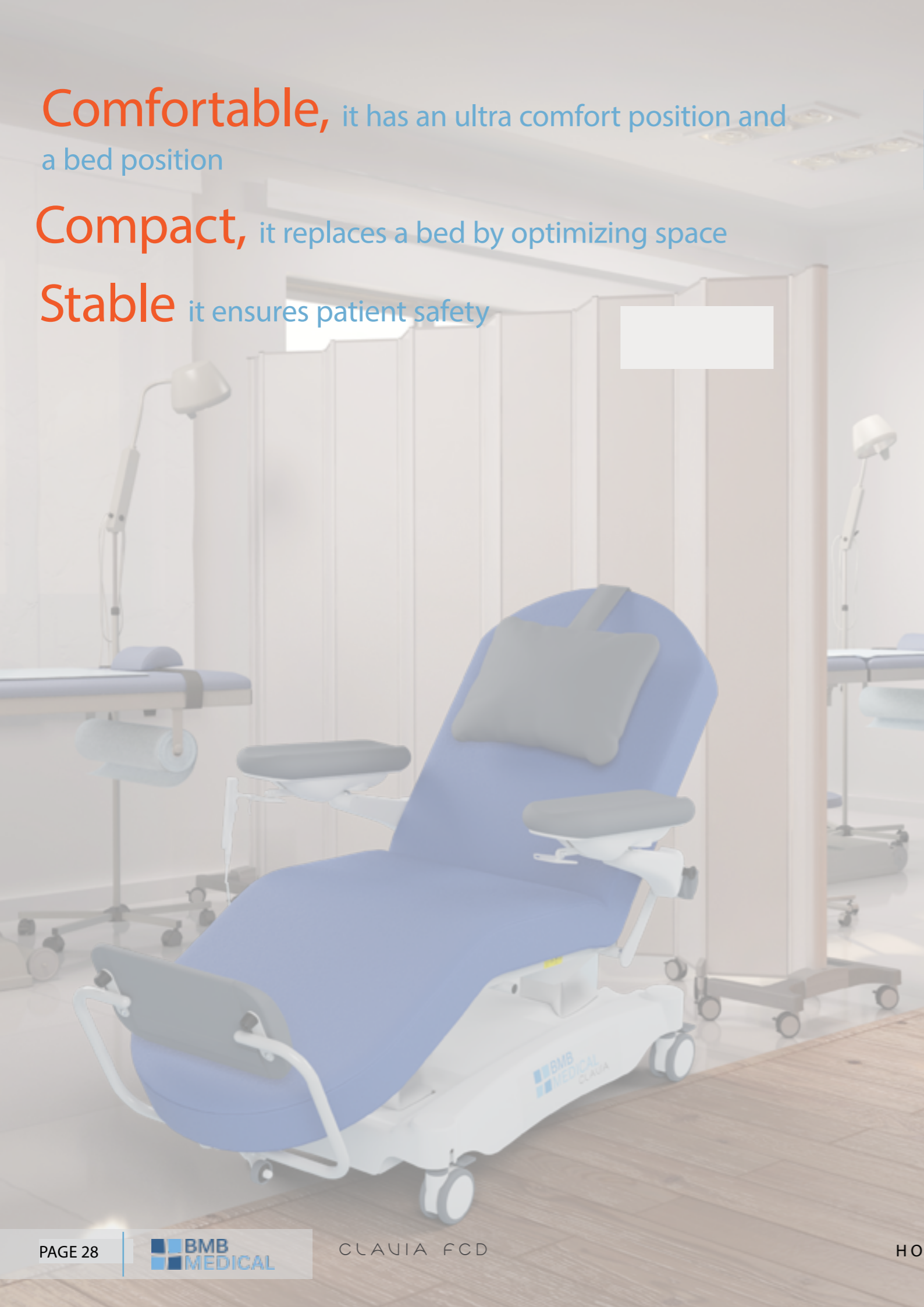

## CLAVIA FCD CHEMOTHERAPY CHAIR

CLAVIA FCD is a chemotherapy / dialysis chair with electric movements.

Comfortable, it allows patient management during a lengthy care procedure.

Equipped with 6 pre-programmed positions, it is quick and intuitive to use.



# CLAVIA FCD

## CLAVIA FCD DIMENSIONS

	LENGTH	WIDTH	HEIGHT
OVERALL	148 cm chair position 202 cm bed position	94 or 104 cm (depending on model)	59 to 99 cm (with mattress)
MATTRESS BASE	65 x 200 cm or 75 x 200 cm		

PRODUCT WEIGHT : 90 - 92 Kg (depending on model)  
SAFE WORKING LOAD: 250 kg



NON-CONTRACTUAL PHOTOS - PRODUCTS FITTED WITH ACCESSORIES

CLAVIA FCD is a Chemotherapy & Dialysis treatment stretcher bed with electric movements.


Combining comfort, ergonomics and aesthetics, our products are designed to accommodate and meet the needs of patients and caregivers.

### CLAVIA FCD Chemotherapy & Dialysis treatment stretcher bed with electric movements

- Electrically assisted height adjustment on 2 LINAK columns : 49 - 89 cm (without mattress 10 cm thick)
- Electric Trendelenburg (15°). Reverse Trendelenburg (6°)
- Electrically (LINAK) assisted leg rest 0 - 60°
- Electrically (LINAK) assisted backrest, manual release.
- 3-sectional mattress base : 65 x 200 cm or 75 x 200 cm.
- Fixed mattress made of high resilience foam (35 kg/m<sup>3</sup>) - Thickness 10 cm.
- Adjustable headrest.
- Digital actuators allowing 6 programmed positions: Welcome, Relax, Reverse Trend, Secured Trendelenburg position, CPR.
- Machine-welded steel frame, epoxy coated.
- Frame protection cover.
- 4 lockable castors Ø 125 mm.
- Anatomic shaped armrest following the backrest inclination. Adjustable in height (2 positions) and in all directions. Retractable behind the backrest for lateral transfer of the patient.
- 2 holders on siderails for IV pole.



6 programmed positions:  
Welcome, Relax, Reverse Trend,  
Secured Trendelenburg position,  
CPR

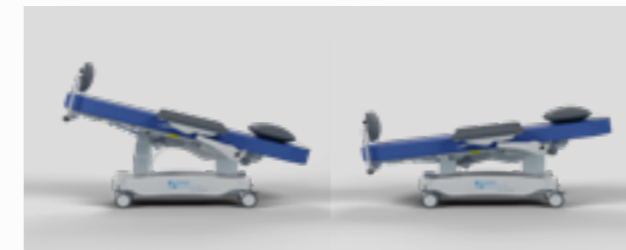

Electrically Trendelenburg (15°)  
Reverse Trendelenburg (6°)



4 lockable castors Ø 125 mm



Adjustable headrest



Anatomic shaped armrest






Anti-static system  
CLORA




Castors Ø 125 mm  
with central brake  
CLORS1

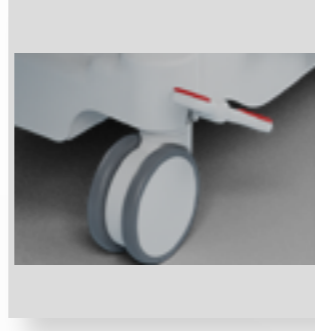


Double running castors Ø 150  
mm with central brake  
CLORS2




5th wheel  
CLO5E

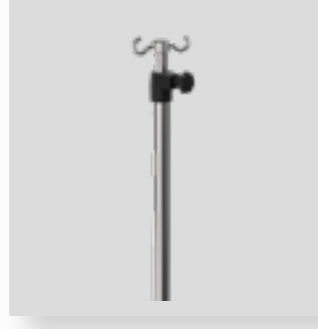


Removable IV pole, 2-stage, 2  
hooks  
TSR2

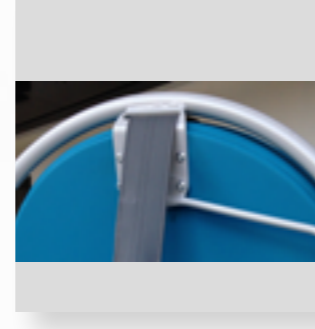


Fixed push bars head-end  
CLOBPT

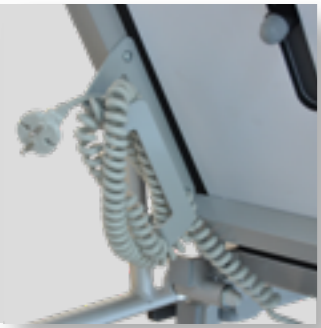

Cable reel with remote holder  
CLOEC

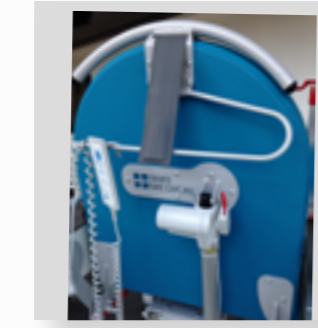


Paper roll dispenser at the head-  
end  
CLOSR

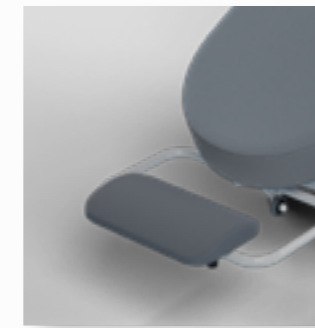

Adjustable electric footrest  
CLORPE



Support for meal tray  
CLOSPR-G  
CLOSPR-D

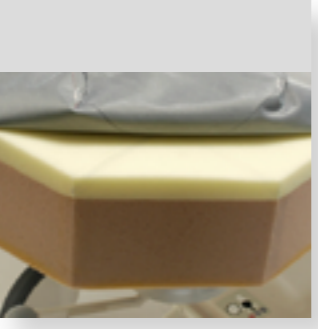

High Comfort mattress  
CLGMGC



Fixed I-Pad support  
with antitheft device  
SUPTAB


## ACCESSORIES

REFERENCE	DESCRIPTION
CLOBA	Battery (50 cycles)
CLO5E	5th wheel (CLORS2 necessary)
CLOEC	Cable reel with remote holder
CLOPR-P	Small Shelf : 42 x 32 cm (CLOSPR-G or CLOSPR-D required)
CLOPR-G	Large Shelf : 81 x 41 cm (CLOSPR-G or CLOSPR-D required)
CLOSR	Paper roll dispenser
CLOBPT	Foldable push bars head-end
CLOSPR-D	Support for meal tray on the right side of the patient (compatible with the CLOSPR-G)
CLOSPR-G	Support for meal tray on the left side of the patient (compatible with CLOSPR-D)
CLOPE	Adjustable electric footrest
CLOBA	Battery (50 cycles)
CLOSR	Paper roll dispenser
CLO5E	5th wheel (CLORS2 necessary)
CLOBPT	Foldable push bars head-end
CLOEC	Cable reel with remote holder
CLOPR-P	Small Shelf : 42 x 32 cm (CLOSPR-G or CLOSPR-D required)
CLOPR-G	Large Shelf : 81 x 41 cm (CLOSPR-G or CLOSPR-D required)
CLOSPR-D	Support for meal tray on the right side of the patient (compatible with the CLOSPR-G)
CLOSPR-G	Support for meal tray on the left side of the patient (compatible with CLOSPR-D)
CLOPE	Adjustable electric footrest
PROTEC RJ	Removable Crystal protection for leg rest
PROTEC RP	Removable Crystal protection for footrest
CLGMGC	High Comfort mattress, high quality foam (35 kg/m3) + 3 cm thick viscoelastic layer (total height : 13 cm)
SUPTAB	Fixed Ipad support with antitheft device
PSU	Attachment hooks for bags on both sides below the patient surface
EQUI	Equipotential connection
TSR2	Removable 2-stage IV pole, 2 hooks
TSR4	Removable IV pole, 2-stage, 4 hooks


# URBAN TEXTILE M1




ABSINTHE

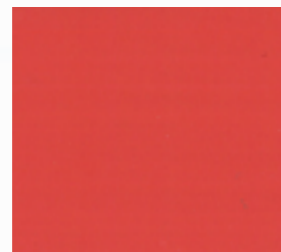


TOMATO

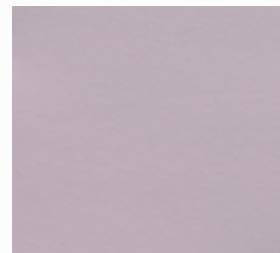


PANSY

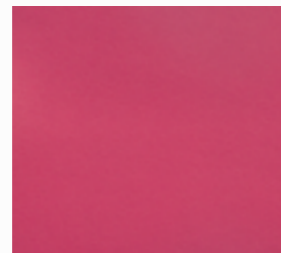

FUCHSIA

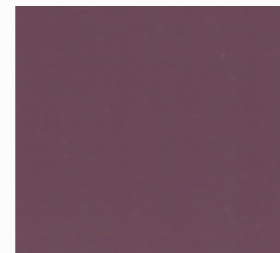


PLUM

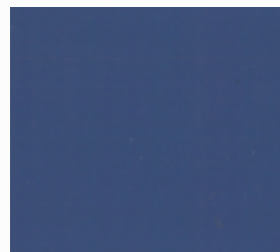

DEEP OCEAN



PETROLE

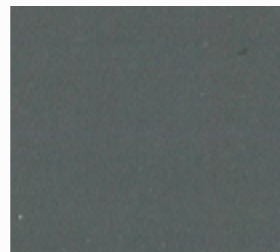

GREY

# COLOR CHART



## CLASSIFICATION

M1


Fine grain, mat and gentle to the touch.  
Rot-proof cotton base.

Composition : Mass-processed PVC 86%, Cotton 14%.

Thickness : 1,04 mm +/- 10%



Effective against  
bacteria(MRSA)

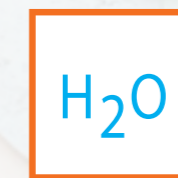

Solvent-free



Phtalate-free



Martindale abrasion  
1000000 cycles



Waterproof

**Comfortable,** it has an ultra comfort position and a bed position

**Compact,** it replaces a bed by optimizing space

**Stable** it ensures patient safety



## CLAVIA FSP MULTIPURPOSE CARE CHAIR

CLAVIA FSP is a versatile treatment chair with electric movements.

Comfortable and easy to handle, it is intended for care services or outpatient examinations.

Lightweight, it simplifies patient transfer.

Equipped with 6 pre-programmed positions, it is quick and intuitive to use.



# CLAVIA FSP

## CLAVIA FSP DIMENSIONS

	LENGTH	WIDTH	HEIGHT
OVERALL	148 cm chair position 208 cm bed position	80 or 90 cm (depending on model)	59 to 99 cm (with mattress)
MATTRESS BASE	65 x 200 cm or 75 x 200 cm		

PRODUCT WEIGHT : 93 - 96 Kg (depending on model)  
SAFE WORKING LOAD: 250 kg



NON-CONTRACTUAL PHOTOS - PRODUCTS FITTED WITH ACCESSORIES

CLAVIA FSP is a multi purpose electric care chair.

Combining comfort, ergonomics and aesthetics, our products are designed to accommodate and meet the needs of patients and caregivers.

### CLAVIA FSP Multipurpose stretcher bed

- Electrically assisted height adjustment on 2 LINAK columns : 49 - 89 cm (without mattress 10 cm thick)
- ElectricTrendelenburg (15°).Reverse Trendelenburg (6°)
- Electrically (LINAK) assisted leg rest 0 - 60°
- Electrically (LINAK) assisted backrest, manual release
- 3-sectional mattress base : 65 x 200 cm or 75 x 200 cm.
- Fixed mattress made of high resilience foam (35 kg/m3) - Thickness 10 cm
- Adjustable headrest
- Digital actuators allowing 6 programmed positions: Welcome, Relax, Reverse Trend, Secured Trendelenburg position, CPR
- Push bars
- Machine-welded steel frame, epoxy coated
- Frame protection cover
- Castors with central braking system Ø 125 mm, including 2 directional ones
- Drop-down laminated side guards. Length 110 x Height 27cm above mattress
- 2 holders on siderails for IV pole
- Battery




6 programmed positions: Welcome, Relax, Reverse Trend, Secured Trendelenburg position, CPR



Castors with central braking system Ø 125 mm, including 2 directional ones




Adjustable headrest

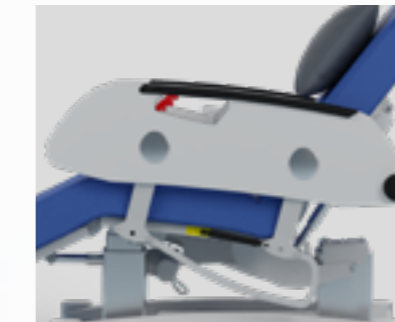

Drop-down laminated side guards



Push bars

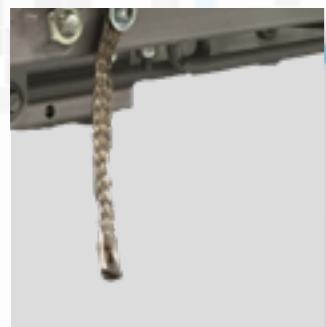


Anti-static system  
CLORA




Double running castors Ø 150 mm with central brake  
CLO150



5th wheel  
CLO5E

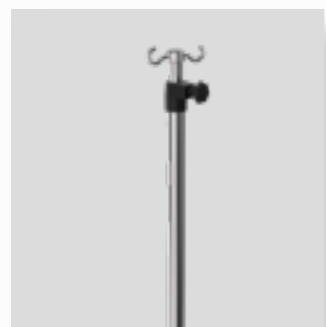

Removable IV pole, 2-stage, 2 hooks  
TSR2

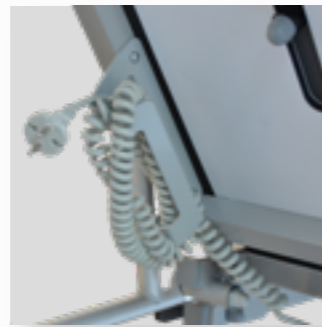

Cable reel with remote holder  
CLOEC



Paper roll dispenser at the head-end  
CLOSR



Adjustable electric footrest  
CLOPE



Support for meal tray  
CLOSPR-G  
CLOSPR-D



High Comfort mattress  
CLGMGC



Attachment hooks for bags on both sides below the patient surface  
PSU



CLAVIA side rails cushion/armrest, removable and right/left reversible (single)  
CLOCB


## ACCESSORIES

REFERENCE	DESCRIPTION
CLORA	Anti-static system
CLO150	Double running wheels Ø 150 mm
CLO5E	5th wheel (CLO150 necessary)
TSR2	Removable 2-stage IV pole, 2 hooks
TSR4	Removable IV pole, 2-stage, 4 hooks
CLOEC	Cable reel with remote holder
CLOSR	Paper roll dispenser for CLAVIA FSP and CLAVIA FCD
CLOPE	Adjustable electric footrest
PROTEC RJ	Removable Crystal protection for leg rest
PROTEC RP	Removable Crystal protection for footrest
CLGMGC	High Comfort mattress, high quality foam (35 kg/m <sup>3</sup> ) + 3 cm thick viscoelastic layer (total height : 13 cm)
PSU	Attachment hooks for bags on both sides below the patient surface
EQUI	Equipotential connection
CLOCB	CLAVIA side rails cushion/armrest, removable and right/left reversible (single)
CLOSPR-G	Support for meal tray on the left side of the patient (compatible with CLOSPR-D)
CLOSPR-D	Support for meal tray on the right side of the patient (compatible with the CLOSPR-G)
CLOPR-P	Small Shelf : 42 x 32 cm (CLOSPR-G or CLOSPR-D required)
CLOPR-G	Large Shelf : 81 x 41 cm (CLOSPR-G or CLOSPR-D required)


# URBAN TEXTILE M1




ABSINTHE

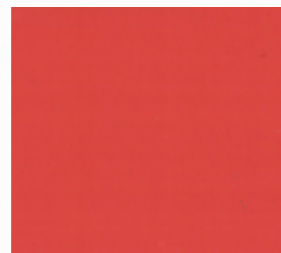


TOMATO

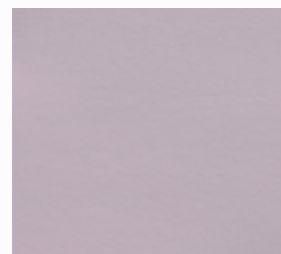


PANSY

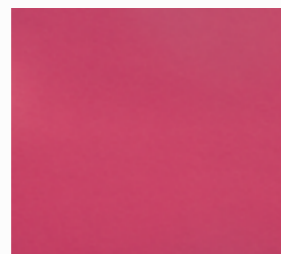

FUCHSIA

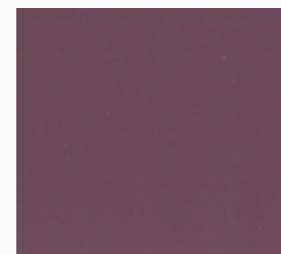


PLUM

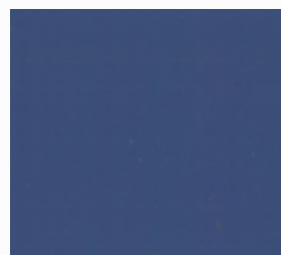

DEEP OCEAN



PETROLE

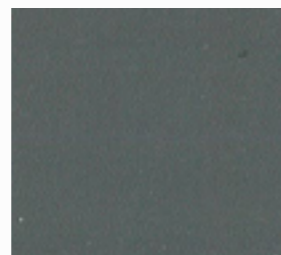

GREY

# COLOR CHART



## CLASSIFICATION


M1

Fine grain, mat and gentle to the touch.  
Rot-proof cotton base.

Composition : Mass-processed PVC 86%, Cotton 14%.  
Thickness : 1,04 mm +/- 10%



Effective against  
bacteria(MRSA)

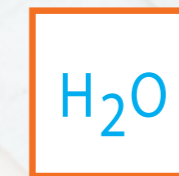

Solvent-free



Phtalate-free



Martindale abrasion  
1000000 cycles



Waterproof

A medical professional must always rely on his or her own professional clinical judgment when deciding whether to use a product(s) when treating a particular patient.

A healthcare professional anyway, refers to the package insert, product label and/or instructions for use before using any BMB MEDICAL product.

The products depicted are CE marked in accordance with EU Regulations and Directives.

BMB MEDICAL – 100 Avenue du Formans – CS 11001 – 01604 TREVOUX CEDEX – FRANCE

Tel : +33 474 08 71 71 – Fax : +33 474 08 71 72 – email : info@bmb-medical.com

Sarl au capital de 600 000 € - RCS Bourg en Bresse 89B652

SIRET : 950 413 229 00036 – Code NAF : 3250A – Code TVA : FR68 950 413 229

[www.bmb-medical.com](http://www.bmb-medical.com)



Tél : +33 (0)4 74 08 71 71  
Fax : +33 (0)4 74 08 71 72  
[info@bmb-medical.com](mailto:info@bmb-medical.com)

100 Av. du Formans - CS 11001  
01604 TREVOUX Cedex  
FRANCE

[www.bmb-medical.com](http://www.bmb-medical.com)