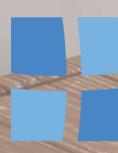
# VESTIAIRE

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## USER MANUAL

 **BMB**  
**MEDICAL**

 [FRANÇAIS](#)  
 [ENGLISH](#)

# FRANÇAIS

---

## BMB MEDICAL

100 Avenue du Formans  
CS 11001  
01604 TRÉVOUX CEDEX  
Tél : +33(0)4 74 08 71 71  
Fax : +33(0)4 74 08 71 72

[www.bmb-medical.com](http://www.bmb-medical.com)

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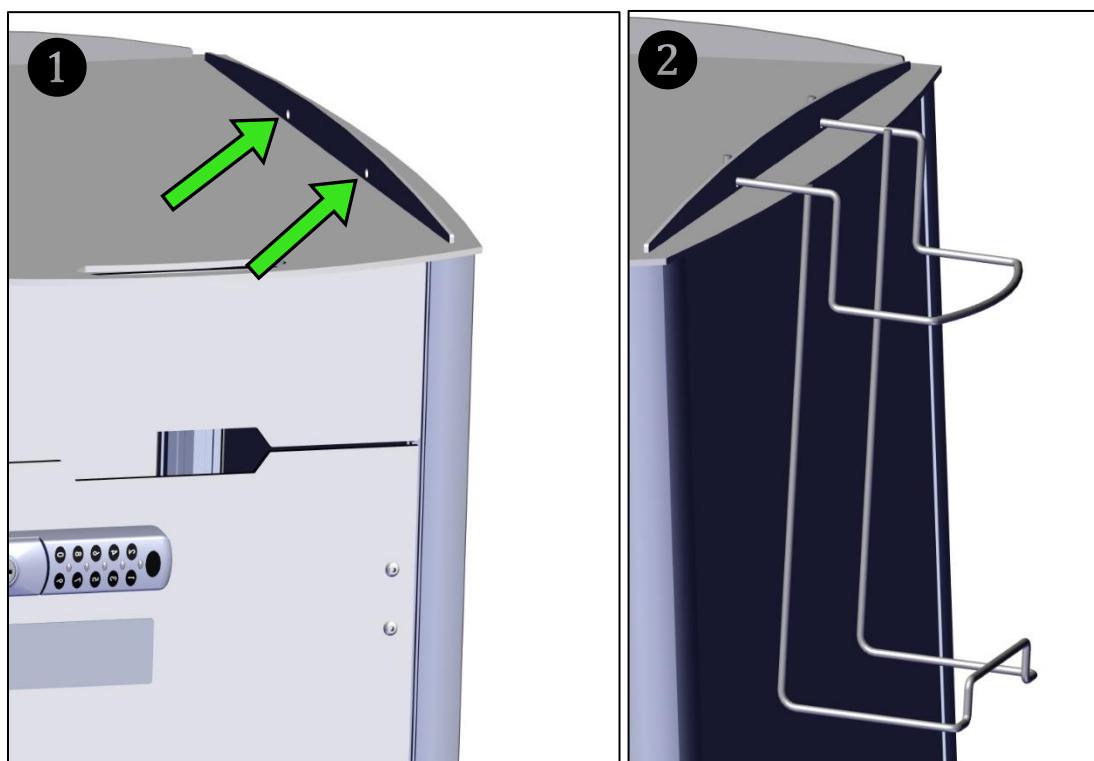
[www.bmb-medical.com](http://www.bmb-medical.com)

This manual is aimed at familiarising you with the use, maintenance and safety of your new medical device. We urge you to read this document carefully and follow its recommendations for optimum safe use.

**BMB MEDICAL** offers a broad variety of options for the products covered by this user manual. Consequently, some features described in this manual may not apply to your product.

Some illustrations are provided for information purposes and do not accurately represent the device in your possession.

Stick the  
Product reference label here

The information in this document are subject to change without notice and in no way engage the responsibility of BMB MEDICAL. This document may not be reproduced, in whole or in part, without prior written consent by BMB MEDICAL. This illicit representation or reproduction, by any means, constitutes counterfeiting. The illustrations in this document are not binding.

# CONTENTS

---

1	INTRODUCTION SPECIFICATIONS .....	17
1.1	PRODUCT PRESENTATION .....	17
1.1.1	BEDSIDE CABINET LOCKER – large version .....	17
1.1.2	BEDSIDE CABINET LOCKER (small version) .....	18
1.2	SCOPE .....	19
1.3	PURPOSE OF THE USER MANUAL.....	19
1.4	STORAGE.....	19
1.5	SYMBOLS AND STANDARDS USED .....	19
2	INSTRUCTIONS FOR USE .....	20
2.1	KEYED LOCKING SYSTEM .....	20
2.2	DIGITAL LOCKING SYSTEM.....	20
1.1.1	PRODUCT BRAKING .....	22
3	CLEANING AND DISINFECTION .....	23
3.1	INTRODUCTION .....	23
3.2	PERIODICITY .....	23
3.3	RECOMMENDED EQUIPMENT AND PRODUCTS .....	23
3.4	CLEANING AND DISINFECTION ADVICE.....	24
3.4.1	CLEANING DEPENDING ON ELEMENTS.....	24
3.4.2	INSTRUCTIONS TO FOLLOW .....	24
4	WARNING .....	25
4.1	ABNORMAL USES.....	25
4.2	TRANSPORT AND STORAGE .....	25
4.2.1	TRANSPORT .....	25
4.2.2	STORAGE .....	26
4.3	WARRANTY .....	26
4.4	AFTER-SALES SERVICE .....	27
4.5	ENVIRONMENT.....	27
4.5.1	USED EQUIPMENT.....	27

# 1 INTRODUCTION SPECIFICATIONS

---

## 1.1 PRODUCT PRESENTATION

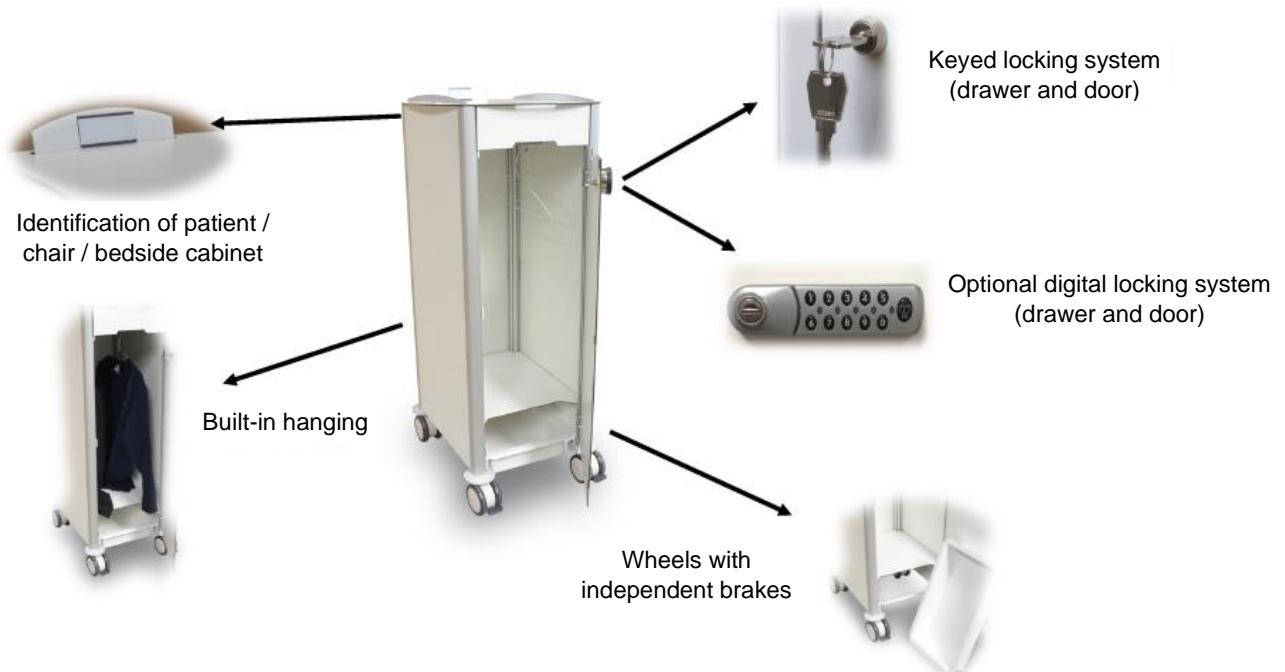
### 1.1.1 BEDSIDE CABINET LOCKER – large version

TECHNICAL DATA	
<b>REFERENCE</b>	LOCKER
<b>DESIGNATION</b>	DAY SURGERY LOCKER – large version
<b>SAFE WORKING LOAD</b>	52 kg (indicates the maximum admissible load including the total weight of the patient and accessories)
<b>WEIGHT</b>	25 kg
<b>OVERALL DIMENSIONS</b>	Overall width: 47.5 cm Overall length: 59 cm Overall height: 126 cm
<b>EQUIPMENT</b>	Stratified compact, acrylic resin finishing touch, white color 4 double running wheel castors Ø 100mm Removable shoes tray in white ABS Key lock system (2 keys included) Push handle - Drawer in ABS - Wardrobe for clothes Identification system for patient / care chair / locker
<b>AVAILABLE OPTIONS</b>	Universal key for key lock system Digital door lock with code (1 universal key included) Additional universal key for digital lock

### 1.1.2 BEDSIDE CABINET LOCKER (small version)

TECHNICAL DATA	
<b>REFERENCE</b>	LOCKER
<b>DESIGNATION</b>	DAY SURGERY LOCKER – small version
<b>SAFE WORKING LOAD</b>	52 kg (indicates the maximum admissible load including the total weight of the patient and accessories)
<b>WEIGHT</b>	25 kg
<b>OVERALL DIMENSIONS</b>	Overall width: 47.5 cm Overall length: 59 cm Overall height: 92 cm
<b>EQUIPMENT</b>	Stratified compact, acrylic resin finishing touch, white color 4 double running wheel castors Ø 80mm Removable shoes tray in white ABS Key lock system (2 keys included) Push handle - Drawer in ABS - Wardrobe for clothes Identification system for patient / care chair / locker
<b>AVAILABLE OPTIONS</b>	Universal key for key lock system Digital door lock with code (1 universal key included) Additional universal key for digital lock Garbage basket for garbage bag 10L

### OVERVIEW

## **1.2 SCOPE**

The beside cabinet / locker is the ideal product to welcome your patient in a day surgery unit. It is equipped with a secured locker for the storage of your patients' personal effects.

## **1.3 PURPOSE OF THE USER MANUAL**

Healthcare professionals, this user manual was created for you to provide you with the necessary information and precautions to observe for autonomous and safe use.

The product's service life is dependent upon observance of the maintenance and tracking instructions.

It is thus essential that all staff has access to and be informed of the handling procedures and risks associated with the use of this device. The latter should be handled by a qualified person who is aware of the risks inherent to improper use. Additional training is not required as we consider that knowledge of the user manual is sufficient.

## **1.4 STORAGE**

This user manual should be kept near the product, in a place provided for this purpose. It must be shielded from all elements and substance that could compromise its perfect legibility.

## **1.5 SYMBOLS AND STANDARDS USED**

This is a signal that not following the recommendation may endanger the patient or user, or damage the equipment.

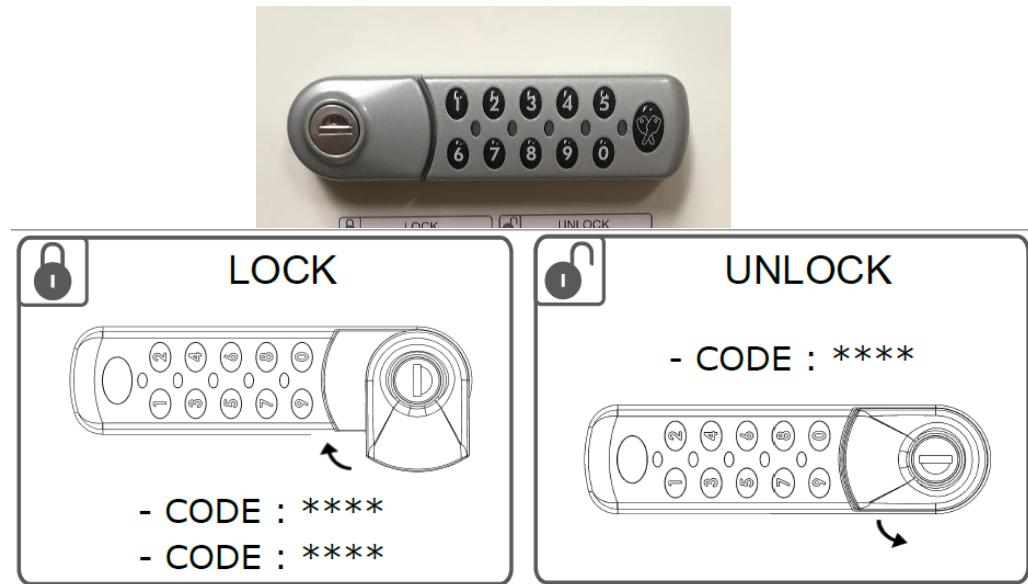
## 2 INSTRUCTIONS FOR USE

### 2.1 KEYED LOCKING SYSTEM

Every locker is sold with two unique keys. These two keys are linked to a single locker. A universal key may be ordered as an optional feature.

### 2.2 DIGITAL LOCKING SYSTEM

#### In order to lock the door:

Enter your 4-character code twice.

#### In order to unlock the door:

Enter the code once.

In case you forget your code, the master code is: **22446688**

Batteries must be checked regularly.

### ETAT DES PILES / BATTERIES

Press x2

Si rouge changer pile  
*If red change battery*

Autonomie (Vert:1 à 4)  
*Autonomy (Green:1 to 4)*

If the red light flashes, the procedure to change the batteries is as follows:

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The system is equipped with two **AALR6 Alkaline 1.5V** batteries.

### 1.1.1 PRODUCT BRAKING

**Except during transport, please keep the locker in brake position.**

Check the brakes of the product by trying to move the locker. Do not park the product on slanting floor.

Wheels in non-brake position

Wheels in brake position

## 3 CLEANING AND DISINFECTION

---

### 3.1 INTRODUCTION

Regular and thorough cleaning will guarantee the device's life (see cleaning program drawn up at the time of device commissioning, designating the reference staff).

**Failure to perform these cleaning operations can lead to a risk of infection caused by the presence of secretions and/or residues on the chair.**

The following cleaning advice is not intended to replace your establishment's in-house protocol.

### 3.2 PERIODICITY

The cleaning must be done:

- At each patient, i.e. between two patients.
- Entirely every week

### 3.3 RECOMMENDED EQUIPMENT AND PRODUCTS

Use only authorized and standardized cleaning products or disinfectants.

Disinfectant molecules adapted to the furniture:

- Aldehydes
- Formaldehydes
- Glutaraldehydes
- Alcohols
- Phenols
- Chlorhexidine
- Amphoteric active surfactant
- Quaternary ammoniums

Familiarize yourself with products that are strictly prohibited by the manufacturer (see list below).

**List of strictly prohibited products:**

<b>Solvents:</b>	Essence / superior aliphatic and aromatic alcohols / ketone esters / trichlorethylene-perchlorethylene
<b>Base:</b>	Soda 10% / Ammonia 10%
<b>Acids:</b>	Nitric Acid 10% / Hydrochloric Acid 30% / Citric Acid 10%
<b>Miscellaneous:</b>	Distilled water / Pure Javel or slightly diluted water / Abrasive powders /Any abrasive product

**NB:**

- Quaternary cleaners contain ingredients with the “-yl” suffix and ammonium chloride.
- Quaternary/Isopropyl cleaners contain one quaternary ingredient as listed above, as well as isopropyl alcohol
- Phenolic cleaners contain ingredients with the “-phenol” suffix
- Javel water is known by its generic name “sodium hypochlorite”

- Do not use abrasive utensils or undiluted bleach.
- Do not mix the products.
- Respect the prescribed dosages and dilutions.
- Use clean, single-use wipes.
- Clean and disinfect with protective clothing and gloves.

**High-pressure cleaning and steaming are prohibited.**

**BMB medical shall not be held liable for any damage resulting from non-compliance with operating instructions of a disinfecting detergent product under the warranty.**

### **3.4 CLEANING AND DISINFECTION ADVICE**

#### **3.4.1 CLEANING DEPENDING ON ELEMENTS**

After cleaning, dry the locker and check its proper operation.

#### **3.4.2 INSTRUCTIONS TO FOLLOW**

All furniture surfaces must be cleaned.

During cleaning or disinfection, it is recommended to:

- Spray or apply the product by means of a cloth: Wash with running water and soap or diluted alcohol.
- Start with the areas closest to the patient by insisting on parts in contact with users.
- Start from the top down.

Leave the equipment to dry.

Products that cause stains (e.g.: “blue product” or eosin) must be cleaned as rapidly as possible to prevent them from soaking in.

## 4 WARNING

---

### 4.1 ABNORMAL USES

**This locker is designed for and adapted to its field of application. Any misuse may cause accidents for the user or even irretrievable breakdown..**

The following list of misuse examples of the lockers is not comprehensive and BMB MEDICAL appeals to users' common sense.

- Do not sit on the locker.
- Do not use the locker with any load over 15 kg.
- Do not park the locker on a floor with a slant over 5°.
- Do not move the locker by pulling it from the door or drawer.
- Always move the locker with the door and drawer locked in closed position.
- Do not move the locker with a load on the tray.
- Do not wash the locker with a high-pressure jet (see cleaning warning).
- Or any other use unrelated to the locker's normal operation.

### 4.2 TRANSPORT AND STORAGE

Transport and storage conditions:

Ambient conditions	Operation
<b>Ambient temperature</b>	-10 to +50°C
<b>Relative humidity</b>	20 to 90% at 30° C without condensation
<b>Atmospheric pressure</b>	700 to 1,060 ha

#### 4.2.1 TRANSPORT

While transporting a locker for storage, we recommend the following to prevent any risks of impact or fall:

- To strap it to a support in the means of transport
- To cover it
- To set it in brake position

Damage caused during transport is not covered by the warranty. The repair and substitution of damaged parts will be at the customer's expense.

## **4.2.2 STORAGE**

When the locker is stored, make sure that it is:

- In brake position
- Covered

The product must be stored in a clean, well-ventilated, non-humid location, and kept away from bad weather and direct sunlight.

During storage, no heavy loads must be placed on top.

The locker must not be considered a resting place for any type of equipment.

The device must not be exposed to, let alone in contact with, any combustion heat sources and flammable agents.

## **4.3 WARRANTY**

*BMB MEDICAL grants its customers a commercial warranty of two (2) years from the date of delivery, within the limits defined below.*

*This commercial warranty is not intended to replace the legal guarantee of conformity mentioned in articles L.211-4 to L.211-13 of the French Consumer code, or that pertaining defects of the purchased object, covered by articles 1641 to 1648 and 2232 of the French Code of civil law.*

*Beyond the legal guarantee, BMB MEDICAL guarantees the exchange and replacement of parts accepted to be out of order, due to machining fault, assembly fault or material failure. The scope of the warranty does not include parts made unusable either by normal wear, or by improper use or non-compliant repair.*

*Where applicable, at the request of BMB MEDICAL, any product or part that is the subject of a warranty claim must be returned postage paid to the BMB MEDICAL plant. Any improper use, alteration or repair performed with parts not supplied or certified by BMB MEDICAL will void this warranty.*

*BMB MEDICAL products are designed for an expected service life of 10 years under normal conditions of use and with appropriate periodic maintenance as described above. BMB MEDICAL guarantees the original buyer that the welds on its products will be free from structural defects during the expected 10-year service life of the product as long as the original buyer is in possession of the product.*

*No guarantees are provided concerning appropriateness for a specific use.*

*This warranty excludes any disposable parts, IV poles, mattresses, batteries, along with damage incurred by abuse.*

The product warranty shall be considered expired, however, by any of the following:

- Abnormal use (*Warning paragraph*)
- Work without prior notice to our technical services, or maintenance on:
  - A mechanical part of the product
- Failure to observe any other term of the warranty
- Any inappropriate washing
- The use of accessories not provided by the manufacturer
- Failure to comply with the recommended preventive maintenance operations.

#### 4.4 AFTER-SALES SERVICE

For any information concerning product installation, use and maintenance, the technical documentation (wiring diagrams, list of components, maintenance procedures, etc.), or any other questions, please contact the After-Sales Service at the following address: **sav@bmb-medical.com, or by phone at +33 (0)4 74 08 71 71, or your local distributor for overseas purchases.**

From your "Customer Area" on the website [www.bmb-medical.com](http://www.bmb-medical.com), you can access your equipment, submit work requests and track these requests directly via this secure interface.

The operator must carefully read the information contained in this user manual, in particular the information concerning the precautions relating to safety and installation/use methods.

In the event of doubt concerning the correct interpretation of the instructions, do not hesitate to contact the After-Sales Service whose staff are always available.

To facilitate ordering spare parts, you can request the exploded diagrams of your products from the After-Sales Service. You can thus obtain the references of the desired parts.

#### 4.5 ENVIRONMENT

##### 4.5.1 USED EQUIPMENT

The end-of-life equipment must enter the appropriate recovery circuit (waste collection center) for recycling.

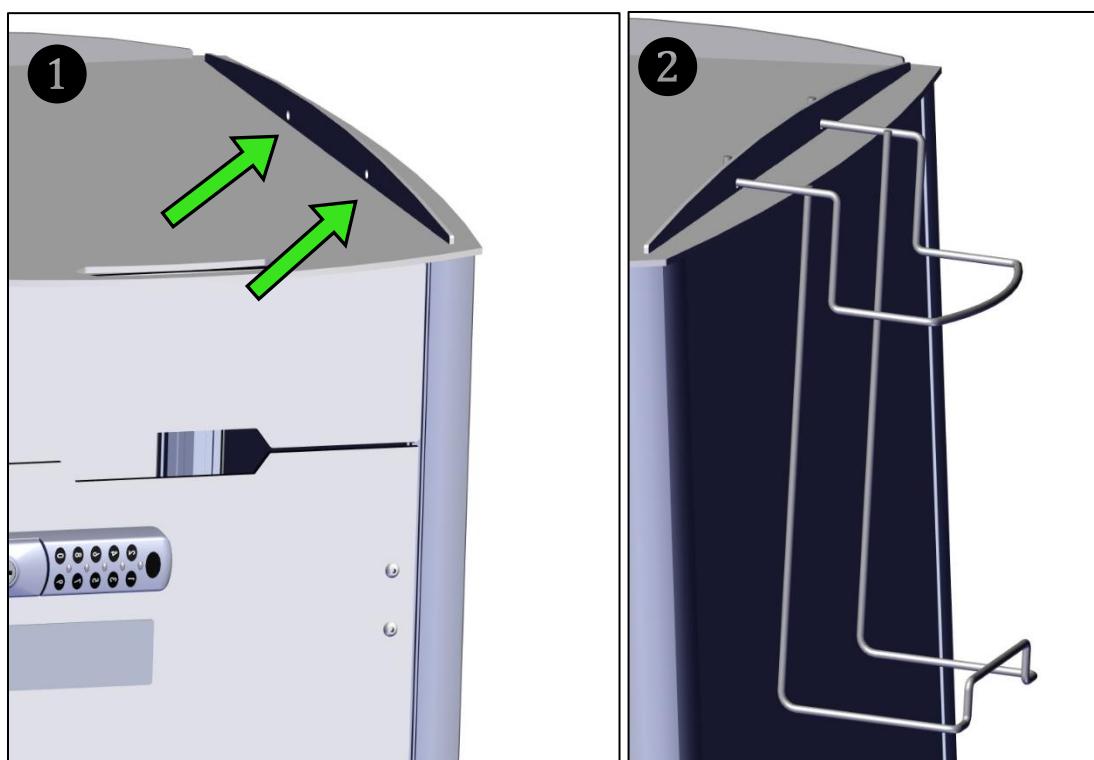
The easy dismantling of the locker allows its elimination and its recovery according to the regulations in force.

## 5 OPTIONS

### 5.1 WASTE BAG-HOLDER

Two entries are provided (image 1) to insert the waste basket on the locker (image 2)

The basket is compatible with a 10L waste bag

Tél: +33 (0)4 74 08 71 71

Fax: +33 (0)4 74 08 71 72

[info@bmb-medical.com](mailto:info@bmb-medical.com)

100 Av. du Formans – CS 11001  
01604 TREVOUX Cedex  
FRANCE